ADVANCED  
BIONICS

POWERFUL CONNECTIONS

# Naída™ CI M90 Sound Processor User Guide

*English, Français, Español, Deutsch, Nederlands, Italiano*

# Symbols Table

## Labeling symbols and their meanings:

	European Community Mark of Conformity. Authorized to affix the CE Mark in 2021
	Date of Manufacture
	Manufacturer
	Model number
	Serial number
	LOT number
	European authorized representative
	Unique Device Identifier
	Medical Device
	Type of Protection: BF
	Suitable temperature range for transport and storage
	See Instructions for Use
	Fragile
	Suitable relative humidity range for transport and storage
	Caution (refer to “Cautions & Warnings” section for further information)
	Dispose of in accordance with applicable national and local regulations
	MR Unsafe
	Ingress Protection Rating

# Table of Contents

<b>Precautions, Cautions, and Warnings .....</b>	<b>3</b>
<b>Introduction.....</b>	<b>9</b>
<b>The Naída CI M90 Sound Processor .....</b>	<b>9</b>
<i>Compatibility Information .....</i>	<i>10</i>
<i>How to Use your Naída CI M90 Sound Processor .....</i>	<i>11</i>
<b>Information about Naída CI M90 Accessories .....</b>	<b>26</b>
<i>M Batteries and M Battery Charger.....</i>	<i>26</i>
<i>Slim HP, Slim HP Mic, and Slim HP AquaMic.....</i>	<i>29</i>
<i>M Zn Air Battery Pak .....</i>	<i>32</i>
<i>M Waterproof Battery .....</i>	<i>39</i>
<i>M Wearing Clip, M Snuggie, and M Retention Cuff.....</i>	<i>47</i>
<i>M Listening Check.....</i>	<i>52</i>
<i>AB Remote Mobile Application .....</i>	<i>57</i>
<b>Caring for Naída CI M90 and Accessories .....</b>	<b>58</b>
<b>Disposing of Naída CI M90 and Accessories.....</b>	<b>59</b>

## Precautions, Cautions, and Warnings

- This device should be used only by the individual for whom it is prescribed.
- **CHOKING HAZARD:** Contains small parts that pose a hazard of inhalation, choking, or ingestion. Using your sound processor and accessories contradictory to their intended use (e.g. mouthing, chewing) may cause bodily harm. If any parts are swallowed or inhaled, consult a physician or hospital immediately.
- Ensure appropriate supervision when child is wearing the Naída™ CI M90 sound processor and accessories. Do not allow children to play with or leave them unattended with the sound processor or any of its accessories.
- Power supplies and battery chargers should be operated in an open area to ensure adequate airflow. While no injury cases have resulted, components may become hot during normal use or a fault condition. If the device's temperature results in discomfort or pain when touched, disconnect the power source and contact your local Advanced Bionics representative.
- Do not use any other charger or power supply with the sound processor or accessories unless it is supplied by Advanced Bionics. If needed, please call Advanced Bionics for a power supply replacement.
- Do not use accessories when they are plugged into power sources such as wall outlets, or other power sources that are USB compatible (e.g. laptops).
- Do not allow leaking battery fluid to come into contact with skin, mouth, or eyes.
- Do not expose batteries to heat (e.g., do not store in direct sunlight or in a hot car).
- Remove batteries from your sound processor when they are drained to prevent damage from possible leaking. If a battery shows visible signs of leakage, dispose of it in accordance with local regulations.
- Do not dispose of batteries in fire.
- Do not immerse M Batteries or the M Battery Charger directly in water.
- Carry M Batteries in the carrying case provided or in a sealed plastic bag. M Batteries can short circuit if they come into contact with metallic objects such as coins or keys.
- High levels of static electricity may damage electronic components

of your sound processor or implant. Care should be taken to avoid exposure of the system to static electricity.

- Do not attempt to pair your sound processor to any accessories while driving or operating heavy machinery.
- Pair the sound processor to Bluetooth devices only in a secure environment, such as at home or in the hearing care professional's office.
- Unwanted paired Bluetooth devices can be overwritten by pairing with authorized devices or can be deleted during fitting by your cochlear implant professional.
- Do not stream audio to your sound processor while driving or operating heavy machinery.
- Attenuating ambient sounds or adjusting environmental balance while streaming audio (either with the multifunction button or with the mobile application) may compromise situational awareness.
- Use your sound processor for hands-free phone use only where permitted by law and only when you will not be distracted from the safe operation of your vehicle.
- Do not use your Advanced Bionics wireless accessories when instructed not to use wireless electronic devices, such as on airplanes.
- Remove your sound processor and headpiece before entering a room where an MRI scanner is located and prior to having any surgeries that involve electrocautery and diathermy.
- Remove your sound processor and consult your cochlear implant professional if uncomfortable sounds are heard or in case of discomfort, pain or skin irritation.
- If the sound processor or accessories become unusually hot or warm, discontinue use immediately and contact Advanced Bionics or your cochlear implant professional.
- Do not use a headpiece, magnet, or any associated part, that is broken, damaged, or cracked.
- The color cap must be attached when the headpiece is in use.
- Due to increased risk of strangulation, do not use cables longer than 81cm (32") with young children.
- The large Slim HP Color Cap is for use by recipients older than 3 years of age, as this color cap and the Slim HP magnet may become displaced upon dropping.
- Do not remove or replace the Slim HP cable from the sound processor or waterproof wearing solution while underwater. If the cable becomes accidentally detached during water use, remove the

headpiece and sound processor from the water and dry both devices prior to re-connecting the cable to the sound processor.

- It is important to have the correct magnet strength, so you do not experience discomfort or retention issues. If magnet strength is insufficient, the headpiece may fall off more than is acceptable. If magnet strength is excessive, you may experience irritation or discomfort. Consult your cochlear implant professional if there are any concerns regarding magnet strength. If deemed appropriate, an audiologist may change the strength of the magnet in the headpiece. Do not change magnet strength unless under the direction of a cochlear implant professional. If you experience any redness, irritation, or discomfort, discontinue use of the headpiece immediately and contact a cochlear implant professional. See the headpiece instructions for use for additional information regarding adjustment of headpiece magnet strength.
- Store additional headpieces away from items with magnetic strips (e.g., credit cards, hotel room keys, etc.) as they may de-magnetize cards.
- The magnetic field created by the headpiece magnet may alter the programming of magnetic shunts. Users of magnetic shunts should consider off-the-shelf retention methods.
- The digitally-coded, inductive transmission technology used in this device is extremely reliable and experiences virtually no interference from other devices. It should be noted, however, that when operating the device near a computer terminal or other strong electromagnetic field (e.g. RFID system, metal detectors, electromagnetic anti-theft systems), it may be necessary to be at least 24" (60 cm) away to ensure proper operation. If the Naída CI M90 sound processor does not respond to the implant device because of an unusual field disturbance, move away from the disturbing field.
- The Naída CI M90 sound processors and accessories should be used in accordance with the electromagnetic compatibility information provided in the Guidance and Manufacturer's Declaration section of your instructions for use document.
- Although electromagnetic emissions of your Naída CI M90 sound processor and accessories have been confirmed within safe limits, some other devices could be sensitive to such emissions. If you notice other devices do not behave as expected when your sound processor is nearby, separate the devices.
- Portable and mobile radio frequency (RF) communications equipment, including radios and cellular phones, may affect sound quality of the Naída CI M90 sound processor and accessories;

however, there is no safety hazard associated with such equipment.

- Do not expose any part of the Naída CI M90 sound processors or accessories to extreme heat, such as an oven, microwave, or hair dryer.
- AutoSense OS™ 3.0, Speech in Loud Noise, Fixed Directional mode, UltraZoom, UltraZoom + SNR Boost, and Speech in 360 programs may dampen sounds that are not in front of the recipient.
- Use of WindBlock, EchoBlock, SoundRelax, NoiseBlock, and/or WhistleBlock may affect the quality of sound.
- Do not use any programs other than an off-ear program when utilizing any off-ear wearing configuration (e.g., in a clip, in the M Waterproof Battery).
- NOTE: In the United States, Fixed Directional mode, StereoZoom, UltraZoom, SoundRelax, NoiseBlock, WindBlock, and EchoBlock are approved for use in pediatric recipients 6 years and above who are 1) able to complete objective speech perception testing in order to determine speech performance and 2) able to report a preference for different coding strategies or features.
- In the event that you experience any issues with your product, please contact your cochlear implant professional or the manufacturer. Do not attempt to service or modify the Naída CI or its accessories. Doing so may compromise system performance and will void the manufacturer's warranty. Products should be serviced only at Advanced Bionics and damaged products should be returned to Advanced Bionics.

### ***Electrostatic Discharge (ESD)***

High levels of static electricity may damage electronic components of your sound processor or implant. Care should be taken to avoid exposure of the system to static electricity. ESD is present when you feel a spark jump from your body or your hair stands on-end.

The following are examples of situations where high levels of electrostatic charge may be created:

- Walking, crawling, or sliding on carpets.
- Sliding on plastic slides.
- Exiting an automobile.
- Pulling sweaters on and off.
- Touching TV or computer screens.
- Removing bedding materials.

To reduce the likelihood of electrostatic discharge damage:

- Touch a person or object with your fingers prior to touching your or your child's headpiece, cable, or sound processor.
- Touch metal surfaces with your hand prior to allowing equipment to contact the metal surface.
- This is particularly important on the playground where high levels of static electricity can be generated. It is important to discharge any static electricity by touching a metal surface with your hand prior to allowing the sound processor, cable, or headpiece to come into contact with the metal surface.
- Take care to avoid contact between your cochlear implant equipment and metal surfaces when exiting a vehicle.
- Remove cochlear implant equipment prior to removing clothing that may contain electric charge (e.g. sweaters).
- Do not touch TV or computer screens. If screens are touched, be sure to touch an object other than your implant system to discharge any charge build-up prior to touching any part of your implant system.
- Use fabric softener on clothing and bedding.

### ***Airport Security Metal Detectors***

Metal detectors and security scanners, including full body scanners, will not damage your implant system; however, metal detectors may be activated as you pass through. To avoid hearing any unwanted sound when walking through metal detectors or security scanners, you should reduce the volume on the sound processor or remove your headpiece.

### ***Ultrasonic Sensors***

Ultrasonic sensors, sometimes used in lighting sensors and security systems, will not damage your implant system; however, they may be picked up by the headpiece microphone and lead to distorted sound quality when you operate your sound processor in the immediate vicinity of such a sensor. Additionally, if the ultrasonic sound is of a very high intensity, the processor system microphone may become damaged. To avoid hearing any unwanted sound, you should reduce the volume on your sound processor or remove your headpiece when passing near an ultrasonic sensor (e.g. in entrances to libraries).

### ***X-Ray Machines***

X-ray machines will not damage your sound processor or implant but may damage your Naída CI M90 and headpiece microphones. Avoid

placing any system microphones in any checked or carry-on baggage that is screened with X-ray. During airport security screenings, the sound processor and microphone should be worn through the metal detector or examined by hand.

## Introduction

This user guide is designed to help recipients and caregivers understand the use and care of the Naída CI M90 sound processor. The Naída CI M90 sound processor, built on the Phonak Marvel platform, features technologies exclusively available from Advanced Bionics and Phonak designed for your best possible hearing experience.

The Naída CI M90 provides access to the latest sound processing and connectivity innovations by Advanced Bionics. This guide will provide all of the information needed to use and care for the Naída CI M90 and its accessories. In addition, this guide will provide important safety information. Please read the entire guide prior to using the Naída CI M90.

## The Naída CI M90 Sound Processor

The Naída CI M90 sound processor is one part of a complete cochlear implant system that enables hearing. The other necessary parts of the system are the cochlear implant and headpiece (with cable).

The Naída CI M90 offers AutoSense OS 3.0 technology, which dynamically adapts to every listening environment, automatically adjusting the sound processor to optimize hearing in any listening environment. The Naída CI M90 offers connectivity options that include integrated Bluetooth®, RogerDirect, and compatibility with Phonak wireless accessories.

This section will familiarize you with the Naída CI M90 Sound Processor and Accessories.

The M T-Mic is a microphone positioned at the opening of the ear canal, for the most natural listening experience.

The sound processor microphones capture sound from all directions and features settings to focus on sounds of interest in noisy listening environments.

The multifunction button has several functions which can be programmed by the hearing care professional. The button can function as a volume control and/or a program change depending on the programming.

The multi-color LED provides visual information about the Naída CI M sound processor status, battery life, program position, and sound processor error conditions.

The battery attaches to the sound processor to provide power to the system. There are 4 battery options for on-ear wearing, including three sizes of lithium ion rechargeable M Batteries and the M Zn-Air Battery Pak. The recipient also has the option of wearing the power source completely off the ear by using the M Waterproof Battery. Please refer to the M Battery and M Battery Charger Instructions For Use and the M Waterproof Battery Instructions for Use for more details on these products.

The Slim HP transmits sound to the cochlear implant. The Slim HP contains a Slim HP Magnet that ensures retention between the cochlear implant and the headpiece. Please refer to the Slim HP Instructions for Use for more details about the Slim HP products.

## ***Compatibility Information***

The Naída CI M90 is compatible with the following implant types from Advanced Bionics:

- **CI1**
- **HiRes™ 90K**
- **HiRes™ 90K Advantage**
- **HiRes™ Ultra**
- **HiRes™ Ultra 3D**

The Naída CI M90 is compatible with the following Advanced Bionics products:

- **M Batteries, M Zn-Air Battery Pak, M Waterproof Battery**
- **M T-Mic™ microphone, M Earhook, M Acoustic Earhook**
- **Slim HP, Slim HP Mic, Slim HP AquaMic™ headpiece**

- **M Wearing Clip, M Snuggie™ accessory, M Retention Cuff**
- **M Listening Check™ accessory**

The Naída CI M90 is compatible with the following accessory products:

- **AB Remote mobile application**
- **Phonak Roger transmitters**
- **Phonak RemoteControl**
- **Phonak TV Connector**
- **Phonak PartnerMic™ microphone**
- **Phonak Naída™ Link M hearing instrument**

The Naída CI M90 sound processor is Bluetooth 4.2 certified. Any mobile device that supports Bluetooth, Hands Free Profile (HFP) and/or Advanced Audio Distribution Profile (A2DP) should work with your Naída CI M90 sound processor. Bluetooth compatibility information should be used as a reference guide only, because hardware and software levels and releases may vary, and numerous hardware and software combinations are possible. Advanced Bionics assumes no responsibility for the compatibility; thus, it is the user's responsibility to do product compatibility testing by himself before making purchasing decisions on smartphones and other Bluetooth-enabled devices.

## ***How to Use your Naída CI M90 Sound Processor***

### ***Powering Naída CI M90***

The sound processor is turned on when a charged battery is attached to the sound processor. When the battery is first attached, the orange LED located in the middle of the multifunction button will indicate battery charge.

The startup program is selected by the cochlear implant professional during programming.

To power down the Naída CI M90 sound processor, simply remove the battery.

### **Attaching the Battery**

Hold the Naída CI M90 in one hand. Rotate the headpiece cable upward in order to avoid bumping it upon battery placement.

Align the battery connector with the sound processor housing side of the connector.

Slide the battery onto the sound processor until it clicks into place. Do not force the battery onto the sound processor. The batteries are designed to be inserted in only one direction; applying force may damage the equipment.

### **Removing the Battery**


Hold the Naída CI M90 in one hand. In the other hand, hold the battery. Rotate the headpiece cable upward in order to avoid bumping it upon battery removal.

Firmly slide the battery away from the M T-Mic, M Earhook, or M Acoustic Earhook.

Continue sliding the battery until it separates from the sound processor.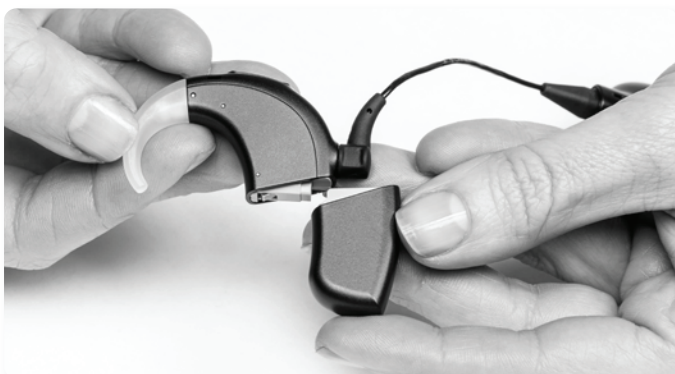

### ***Connecting the earhook to the sound processor***

There are three earhook options for the Naída CI M90 sound processor: the M T-Mic, the M Earhook, and the M Acoustic Earhook.

Beginning with the earhook disconnected and the earhook pin uninstalled, align the earhook flush with the sound processor. Hold the earhook firmly at the top and push towards the sound processor until it meets flush with the sound processor.

Using the provided pin removal tool, place one of the pins in the small hole on the side of the tool. This will help to steady the pin as you line it up with the pin hole on the earhook.


Line up the pin with the hole in the earhook and gently push it through the hole. The pin will move from one side of the sound processor, through the earhook to the other side of the sound processor, holding the earhook in place. The pin should not come out the other side.

If a small portion of the pin extends beyond the sound processor, use the side of the tool to push the pin inwards so it lies flush with the sound processor.

Pull the tool away from the sound processor, leaving the pin inside the sound processor.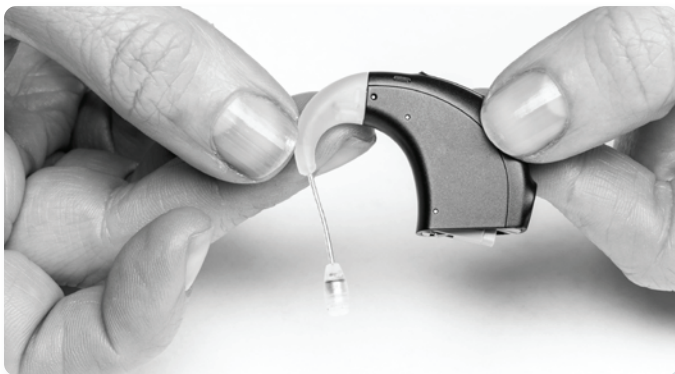

### ***Removing the earhook***

Align the pointed end of the tool with the hole on the sound processor that contains the pin holding the earhook on the sound processor.

Gently press the tool in the hole, pushing the pin out the other side.

The pin does not need to be removed from the sound processor. As long as the tool is pushed in flush on one side of the sound processor, the pin should be displaced enough to adequately remove the earhook. Remove the tool from the sound processor. Gently grasp the earhook at the top near where it connects with the sound processor and pull forward until it is separated from the sound processor.

### ***Attaching the T-Mic cover***

Hold the T-Mic cover. In the other hand, hold the M T-Mic at the end that houses the microphone. Align the bottom of the T-Mic with the T-Mic cover. Gently turn the M T-Mic into the T-Mic cover. Screw until they are flush with one another.

### ***Removing the M T-Mic cover***

Grasp the end of the M T-Mic where the T-Mic cover is located, in one hand. Firmly hold the M T-Mic microphone portion in the other hand. Turn the hand holding the T-Mic cover to slowly unscrew the T-Mic cover from the M T-Mic. Continue turning until the T-Mic cover separates from the M T-Mic.

### ***Attaching the Slim HP cable to the sound processor***

To attach the headpiece cable to the sound processor, hold the headpiece cable by the cable's strain relief. Align the connector on the cable with the connector on the sound processor and gently push the cable into the sound processor until it snaps into place.

### ***Removing the Slim HP cable from the sound processor***

Remove the battery from the processor before removing the cable from the processor. To remove the cable from the sound processor, always hold the cable's strain relief and gently pull it away.

**NOTE:** To maximize cable life, the cable should only be unplugged from the sound processor when changing wearing configuration or when replacing the cable.

### Understanding the LEDs on Naída CI M90

The LED is a programmable feature that provides visual information about the Naída CI M sound processor status, battery life, program position, and sound processor error conditions.

<b>INDICATION</b>	Battery Status at Start-Up
<b>COLOR</b>	Orange
<b>BEHAVIOR</b>	Blinks at start-up with use of rechargeable batteries (not available with M Zn-Air Battery Pak): <ul style="list-style-type: none"> <li>• 4 quick blinks indicate that the battery is fully-charged.</li> <li>• 2 or 3 quick blinks indicate that the battery is sufficiently charged to power the sound processor.</li> <li>• 1 quick blink indicates that the battery is nearly depleted.</li> <li>• No blinking indicates the battery is depleted; replace with a charged or new battery.</li> </ul>
<b>PROGRAMMABLE</b>	No

<b>INDICATION</b>	Low Battery
<b>COLOR</b>	Orange
<b>BEHAVIOR</b>	Continuous Blinking
<b>PROGRAMMABLE</b>	No

<b>INDICATION</b>	Current Program
<b>COLOR</b>	Green

<b>BEHAVIOR</b>	Blinks at start-up after battery status and upon program change <ul style="list-style-type: none"> <li>• 1 long blink is AutoSense program</li> <li>• 1 short blink is program 1</li> <li>• 2 short blinks is program 2</li> <li>• 3 short blinks is program 3</li> <li>• 4 short blinks is program 4</li> </ul>
<b>PROGRAMMABLE</b>	Yes

<b>INDICATION</b>	Flight Mode
<b>COLOR</b>	Orange
<b>BEHAVIOR</b>	One long blink at start-up after battery and program status
<b>PROGRAMMABLE</b>	No

<b>INDICATION</b>	Progressive Level Change
<b>COLOR</b>	Purple
<b>BEHAVIOR</b>	1 blink upon changing
<b>PROGRAMMABLE</b>	Yes

<b>INDICATION</b>	Loud Input
<b>COLOR</b>	Green
<b>BEHAVIOR</b>	Blinks during use in response to loud input sounds. Indicates the sound processor and microphone are responding to sound.
<b>PROGRAMMABLE</b>	Yes

<b>INDICATION</b>	Streaming Input
<b>COLOR</b>	Blue
<b>BEHAVIOR</b>	Blinks while streaming audio to the sound processor. Indicates the sound processor and microphone are streaming audio.
<b>PROGRAMMABLE</b>	Yes

<b>INDICATION</b>	Loss of Lock with the Implant
<b>COLOR</b>	Red
<b>BEHAVIOR</b>	Slow blinking (once per second)
<b>PROGRAMMABLE</b>	Yes

<b>INDICATION</b>	Wrong Implant
<b>COLOR</b>	Red
<b>BEHAVIOR</b>	Fast blinking (more than once per second)
<b>PROGRAMMABLE</b>	No

<b>INDICATION</b>	Error Condition (remove and reattach battery to reset sound processor)
<b>COLOR</b>	Red
<b>BEHAVIOR</b>	Solid Red
<b>PROGRAMMABLE</b>	No

<b>INDICATION</b>	Detect Sound Processor
<b>COLOR</b>	Green
<b>BEHAVIOR</b>	3 blinks upon command from mobile application or fitting software
<b>PROGRAMMABLE</b>	No

### ***Understanding the Internal Alarms on Naída CI M90***

The internal alarms are programmable and provide auditory information about the Naída CI M90 sound processor. Your cochlear implant professional can adjust the volume and pitch of the internal alarms to suit your preference.

<b>INDICATION</b>	<b>BEHAVIOR</b>	<b>PROGRAMMABLE</b>
Low Battery	2 beeps	No
Program Change	Jingle is AutoSense program <ul style="list-style-type: none"> <li>• 1 beep is program 1</li> <li>• 2 beeps is program 2</li> <li>• 3 beeps is program 3</li> <li>• 4 beeps is program 4</li> </ul>	Yes
Volume Change	<ul style="list-style-type: none"> <li>• Long beep at end of range</li> <li>• 2 short beeps at middle of range</li> <li>• 1 short beep at intermediate steps</li> </ul>	Yes
Pairing Successful	Ascending beeps	Yes
TV Connector Available	Ascending beeps	Yes
Phone Ring Tones	Ringling beeps	No

### ***Using the Multifunction on Naída CI M90***

The multifunction button has several functions which can be programmed by the hearing care professional. The button can function as a volume control and/or a program change depending on the programming. Please ask your cochlear implant professional to confirm how your sound processor is programmed.

<b>SITUATION</b>	<b>SHORT PRESS</b>	<b>LONG PRESS</b>
Standard Use	Volume up or down	Program change

SITUATION	SHORT PRESS	LONG PRESS
During Audio Streaming	Streaming volume up or down	Program change
During Phone Calls	Accept phone call (while ringing) or phone call volume up or down (during call)	Reject phone call (while ringing) or end phone call (during call)

## Connectivity Information

The Naída CI M90 sound processor can be connected to Bluetooth-enabled devices for phone calls, VoIP calls, audio streaming, or for use with the AB Remote mobile application.

### Pairing and Connecting the Naída CI M90 to a Bluetooth-Enabled Device

1. Ensure that Bluetooth wireless technology is enabled on your device (e.g., phone, tablet) and search for Bluetooth-enabled devices in the connectivity settings menu.
2. Power on the Naída CI M90 sound processor by connecting the sound processor to a power source. The sound processor will be discoverable by your Bluetooth-enabled device for up to three minutes, or until it is successfully paired to a device.
3. Your device should show a list of Bluetooth-enabled devices. Select the sound processor from the list to pair your sound processor. If you use two Naída CI M90 sound processors, selecting one sound processor for pairing will simultaneously pair both sound processors. A beep confirms successful pairing.
4. The connection will be maintained as long as the device remains on and within range.

After your sound processor has been paired to your Bluetooth-enabled device, the sound processor will automatically connect again when switched on. Your sound processor can be paired to two Bluetooth-enabled devices at a time. The paired sound processor can be unpaired from the Bluetooth device by utilizing the Bluetooth management menu of your device.

***PLEASE NOTE: Once a sound processor has been paired to a device, it is recommended to keep paired devices in silent mode to avoid dropouts related to device notifications.***

## **Managing Bluetooth Phone Calls with the Naída CI M90**

When the sound processor is directly connected with a Bluetooth-enabled phone, you will be able to hear the caller's voice directly through the sound processor. The sound processor picks up your voice through the sound processor microphones.

When making a phone call, you will hear a dialing tone through the sound processor. The sound processor microphones pick up your voice. When receiving a call, a call notification will be heard through the sound processor. You can accept the call with a short press on the upper or lower part of the multifunction button on the sound processor or directly on the phone.

You can reject or end a call with a long press on the upper or lower part of the multifunction button on the sound processor or directly through the phone.

## **Streaming Audio via Bluetooth with the Naída CI M90**

After successful pairing of your processor(s) to your Bluetooth-enabled device, ensure that your device is not muted, and volume is not set to minimum. Then, simply start the audio on the Bluetooth-enabled device and the sound processor will start streaming the audio. Environmental balance and access to surrounding sounds can be managed using the multifunction button, the AB Remote mobile application, or the Phonak RemoteControl.

## **Pairing the Naída CI M90 to Phonak Accessories**

Your Naída CI M90 can be paired to the following Phonak accessories:

- **Phonak RemoteControl**
- **Phonak TV Connector**
- **Phonak PartnerMic**
- **Phonak Roger transmitters**
- **Phonak Naída Link M hearing instrument**

Please refer to the accessory user guide for initial setup and pairing instructions.

## **Switching between Multiple Audio Sources with the Naída CI M90**

A phone call from a Bluetooth-enabled device has highest priority and will pause other audio streaming from a connected device. In order to switch audio streaming source, simply pause the audio from the current source and start streaming from another source.

## ***Placing the Naída CI M90 in Flight Mode***

The Naída CI M90 communicates wirelessly with other devices in the 2.40 GHz to 2.48 GHz frequency range. When flying, some operators require all devices to be switched into flight mode. Entering flight mode will not disable normal sound processor functionality but will disable Bluetooth connectivity functions.

### Activating and Deactivating Flight Mode

1. Hold down either the top or bottom of the multifunction button while attaching the battery.
2. Continue to hold down the button while the sound processor starts up, until a long, solid orange LED is visible, approximately 10 seconds. Note that the start-up LEDs that indicate the battery life of the rechargeable battery and the program number will be seen before the long, solid orange flight mode LED.
3. Removing and reattaching the battery will take the sound processor out of flight mode.

## **Information about Naída CI M90 Accessories**

### ***M Batteries and M Battery Charger***

#### ***General Charging Information for M Batteries***

- **The M Battery Charger provided with the sound processor is designed to recharge up to three M Batteries simultaneously. Using the supplied power adapter, it takes approximately 3 to 4 hours to fully recharge a depleted M Battery. If the M Battery is stored for prolonged periods of time, it may take approximately 5 to 6 hours to fully recharge the battery. This will vary slightly for different M Battery types.**
- **M Batteries do not need to be fully depleted before charging. A partially charged M Battery may be fully charged in a shorter time period.**
- **You can remove the M Battery from the charger at any time.**
- **You will not damage the M Batteries if they remain on the charger longer than the required charging time or overnight.**
- **It is recommended to use and recharge the M Batteries at least once every 3 months. Please note that M Batteries will decrease in capacity with age, even if not in use. This is normal behavior for all rechargeable batteries and should not be considered a defect.**

## Setting up the charger

### To use the M Battery Charger with a wall outlet for charging:

Place the M Battery Charger on a flat surface.

Connect the micro USB end of the power supply cable into the charger.

Connect the power supply cable to the USB wall adapter.

Plug the wall adapter into the wall outlet.

### To use the M Battery Charger with a USB port for charging:

Place the M Battery Charger on a flat surface.

Connect the micro USB end of the power supply cable into the M Battery Charger.

Insert the USB jack into a USB port (on a computer, in a car, etc.).

## Understanding M Battery Charger LED indicators

The M Battery Charger has different indicator lights that are located at the base of each charging port, as well as where the power cable enters the charger. If the light near the power jack is lit and green in color, it means the charger is connected to a power source and ready to charge the M Batteries. If the light near the power jack is lit and red in color, this indicates a fault condition. \*

The LEDs located at the base of each charging port have 3 color options:

- **Blue: M Battery is charging.**
- **Green: M Battery is fully charged.**
- **Red: Fault condition.\***

*\*Try to reinsert the power cable or M Battery. If the red LED illuminates again, contact your cochlear implant professional.*

The charger has an automatic dimmer. The brightness of the LEDs may vary depending on the brightness of the room in which the charger is placed.

## Inserting the M Batteries into the charger

Locate the slide tracks on the charger. Position the M Battery so that the battery contact is facing down toward the base of the charger and aligned with the tracks on the charger.

Slide the M Battery along the tracks until it engages. The charger is designed so the M Batteries can only be inserted in one direction.

### ***Removing the M Batteries from the charger***

Gently slide the M Battery out of the slide tracks of the charger, toward the front edge of the charger. Do not pull directly up on the battery.

Please refer to the M Battery and M Battery Charger Instructions for Use for additional information.

## ***Slim HP, Slim HP Mic, and Slim HP AquaMic***

### ***Assembly Instructions***

1. To operate your headpiece with your cochlear implant system you will need to install a Slim HP Magnet and attach a Slim HP Color Cap, as follows:
2. Insert the Slim HP Magnet into the magnet well of the headpiece.
3. For recipients using the Standard Magnet, no specific magnet orientation is required.
4. For recipients using the 3D Magnet, the magnet should be inserted so that the arrow on the magnet points up when the headpiece is on the head.
5. Attach a color cap (See “Color Caps” section).
6. Attach the headpiece and cable to your sound processor (See “Cables” section).
7. Then place the bottom side of the headpiece over the internal magnet of your cochlear implant.

***NOTE: For recipients of HiRes Ultra 3D implants, if the headpiece repels or pushes away when brought over the internal device, rotate the headpiece back and forth as you move the headpiece closer towards the implant. Once attraction is felt, adjust the headpiece to the original orientation and place on head.***

***NOTE: For recipients of HiRes Ultra 3D implants, when in a magnetic field or when the headpiece is removed, you may feel the internal components of the magnet vibrate and/or rotate. This movement is expected as the magnets are designed to rotate in the presence of strong magnetic fields. This movement may be noticeable and is not expected to be uncomfortable.***

## **Cables**

Slim HP, Slim HP Mic, and Slim HP AquaMic have cables that are integrated into the headpiece.

### **Attaching the Slim HP cable to the sound processor**

To attach the Slim HP cable to the sound processor, hold the headpiece cable by the cable's strain relief. Align the connector on the cable with the connector on the sound processor and gently push the cable into the sound processor until it snaps into place.

### **Removing the Slim HP cable from the sound processor**

Remove the battery from the processor before removing the cable from the processor. To remove the cable from the sound processor, always hold the cable's strain relief and gently pull it away.

**NOTE:** To maximize cable life, the cable should only be unplugged from the sound processor when changing wearing configuration or when replacing the cable.

## Color Caps

### Attaching the Slim HP Color Cap

Align the Color Cap over the Slim HP and snap it into place.

### Removing the Slim HP Color Cap

The arrow on the underside of the Slim HPs indicates where the Slim HP Color Cap removal tool can be inserted to remove the Color Cap from the Slim HP. Insert the tool as indicated by the arrow to pop the Color Cap off of the Slim HP.

## **Magnets**


It is important for your headpiece to have the correct magnet strength and orientation to avoid discomfort or retention issues. Contact your cochlear implant professional if there are any concerns regarding magnet retention. If deemed appropriate, your cochlear implant professional may replace the headpiece magnet with a stronger or weaker magnet or adjust magnet orientation. If needed, you may use off-the-shelf solutions to improve retention.

Please refer to the Slim HP, Slim HP Mic, and Slim HP AquaMic Instructions for Use for additional information.

## **M Zn Air Battery Pak**

### ***Inserting Zn-Air Disposable Battery Cells (Non-Tamperproof Cartridge)***

Holding the battery cartridge in one hand, gently but firmly push down on the lever on the back of the battery cartridge. This will cause the Zn-Air battery drawer to pop out of the bottom of the cartridge. Pull the battery door out of the cartridge with the other hand.

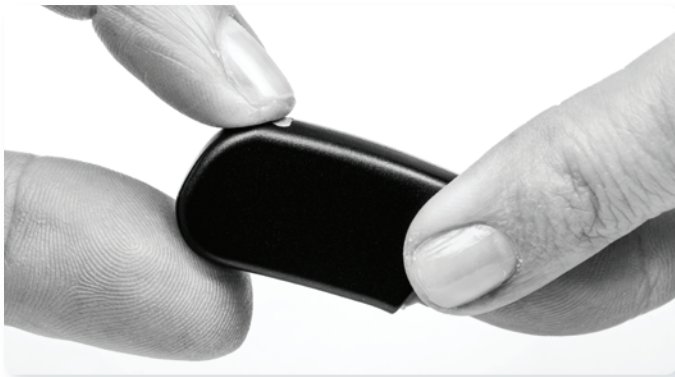Remove the Zn-Air cells from their package, making sure to remove the sticker from the flat positive (+) side of the cell. Removing the sticker allows air to enter and activate the battery, so be sure to use the battery immediately after peeling off the sticker.

Place two Zn-Air cells, positive (flat) side up, in the two openings in the battery drawer.

Once batteries are flush with the top of the battery drawer, gently slide the battery drawer back into the battery cartridge. If the cells are inserted incorrectly, the battery drawer will not close.

Then begin to attach the battery cartridge to the Naída CI by holding the sound processor in one hand. Rotate RF cable upward to avoid bumping it upon battery placement.

Align the battery cartridge connector with the processor housing side of the connector.

Slide the battery onto the processor until you feel it click into place.

### ***Inserting Zn-Air Disposable Battery Cells (Tamperproof Cartridge)***

Using the same tool provided for the M T-Mic or M Earhook removal and replacement, insert the pointed end of the tool into the small hole located on the spine of the Zn-Air cartridge.

With the tool inserted in the hole, gently push in and downward until you feel the Zn-Air drawer release and pop out the button of the Zn-Air cartridge. If you do not have the tool present, any thin, pointed device should work to release the drawer.

Set the tool aside. From here you can grasp the Zn-Air drawer with your hand and gently pull it the rest of the way out, so you are able to insert the Zn-Air cells.

Remove the Zn-Air cells from their package, making sure to remove the sticker from the flat positive (+) side of the cell. Removing the sticker allows air to enter and activate the battery, so be sure to use the battery immediately after peeling off the sticker.

Place two Zn-Air cells, positive (flat) side up, in the two openings in the battery drawer.

Once batteries are flush with the top of the battery drawer, gently slide the battery drawer back into the battery cartridge. If the cells are inserted incorrectly, the battery drawer will not close.

Then begin to attach the battery cartridge to the Naída CI by holding the sound processor in one hand. Rotate RF cable upward to avoid bumping it upon battery placement.

Align the battery cartridge connector with the processor housing side of the connector.

Slide the battery onto the processor until you feel it click into place.

Please refer to the M Zn Air Battery Pak Instructions for Use for additional information.

## ***M Waterproof Battery***

### ***General Information for Using the M Waterproof Battery***

- Using the supplied power adapter, it takes approximately 3 to 4 hours to fully recharge a depleted M Waterproof Battery. If the M Waterproof Battery is stored for prolonged periods of time, it may take approximately 5 to 6 hours to fully recharge the battery.
- The M Waterproof Battery does not need to be fully depleted before charging. A partially charged M Waterproof Battery may be fully charged in a shorter time period.

- You can discontinue charging the M Waterproof Battery at any time.
- You will not damage the M Waterproof Battery if it remains charging longer than the required charging time or overnight.
- It is recommended to use and recharge the M Waterproof Battery at least once every 3 months. Please note that the M Waterproof Battery will decrease in capacity with age, even if not in use. This is normal behavior for all rechargeable batteries and should not be considered a defect.
- Ensure that the selected program supports off-ear listening (headpiece microphone input). If the program does not support headpiece microphone input, select the appropriate program that does. If you are unsure, contact your cochlear implant professional for programming assistance.
- Please note that you may notice a slight sound quality difference when switching between your BTE microphone program and your headpiece microphone program.
- Do not open or close the M Waterproof Battery while in a water environment.
- Before placing the Naída CI M in the M Waterproof Battery, visually inspect the O-Ring seal to make sure it is placed correctly. Confirm that the O-Ring is not dry, brittle or dirty. If it is, replace O-Ring prior to use. If damage to the O-Ring is observed or suspected, replace prior to resuming use.

### ***Inserting the Sound Processor in the M Waterproof Battery***

The M Waterproof Battery has a lock to make sure the case is not accidentally opened, preventing moisture from entering the case and damaging the sound processor. The lock is located on the front of the M Waterproof Battery.

While holding the M Waterproof Battery in one hand, use your forefinger on the opposite hand to push the lock button while using your thumb to pull the lock away from the case.

The M Earhook, M T-Mic, or M Acoustic Earhook, battery, cable, and headpiece must be removed from the sound processor before inserting into the case, however the earhook pin can remain in the processor. Carefully align the battery connector on the sound processor with the battery connector in the M Waterproof Battery and slide the sound processor onto the battery connector within the M Waterproof Battery, connecting the cable connector with the M Waterproof battery to the sound processor.

Slide the sound processor into place until the M Waterproof Battery and headpiece cable connector are coupled.

Once the sound processor is in place, close the lid of the M Waterproof Battery. Return the latch to the closed position, confirming that the latch has clicked into place.

Connect the AquaMic headpiece cable to the M Waterproof Battery.

To attach the clip to the M Waterproof Battery, insert the hinge side of the M Waterproof Battery into the clip and press down until the M Waterproof Battery snaps into place.

To remove the clip, pull the handle of the clip back and away from the M Waterproof Battery.

### ***Removing the Sound Processor in the M Waterproof Battery***

Towel dry the M Waterproof Battery before removing the sound processor to prevent water from accidentally getting on the sound processor. Be careful not to allow water from your hair or body to fall in the M Waterproof Battery or on the sound processor.

While holding the M Waterproof Battery in one hand, use your forefinger on the opposite hand to push the lock button while using your thumb to pull the lock away from the case.

To remove the sound processor, carefully slide the processor forward,

making sure not to pull up until the sound processor is free from the connectors.

Remember to wipe the battery case with a soft, lint-free cloth and air dry overnight after any water related activities. Continue use of the Slim HP AquaMic.

### ***Replacing the O-Ring on the M Waterproof Battery***

The M Waterproof Battery O-Ring should be inspected prior to each use and will need to be changed periodically. If damage to the O-Ring is observed or suspected, replace prior to resuming use. The M Waterproof Battery and O-Ring should be rinsed following submersion in salt or chlorine treated water.

To remove the existing O-Ring, use your fingers to lift a portion of the old O-Ring out of the M Waterproof Battery. Once a portion of the O-Ring is lifted out, hold the O-Ring and pull the remaining portion out of the M Waterproof Battery mounting groove.

To insert the new O-Ring, place a portion of the new O-Ring in the M Waterproof Battery mounting groove on the case. Take care to match the shape of the O-Ring correctly with the shape of the M Waterproof Battery. While holding the O-Ring in the groove with one hand, carefully stretch the remaining portion of the O-Ring into the remaining groove.

Run your thumb over the O-Ring to make sure of proper insertion. Extreme care should be taken to make sure the O-Ring is level and secure in the mounting groove.

### ***Charging the M Waterproof Battery***

#### **To use a wall outlet for charging:**

Place the M Waterproof Battery on a flat surface.

Connect the micro USB end of the power supply cable into the M Waterproof Battery.

Connect the power supply cable to the USB wall adapter.

Plug the wall adapter into the wall outlet.

### **To use a USB port for charging:**

Place the M Waterproof Battery on a flat surface.

Connect the micro USB end of the power supply cable into the M Waterproof Battery.

Insert the USB jack into a USB port (on a computer, in a car, etc.).

Please Note: After use, wipe the M Waterproof Battery with a dry cloth to remove moisture or debris before charging.

***Please Note: You may charge the M Waterproof Battery while the processor is connected, but the sound processor will be powered off during charging. Do not wear the M Waterproof Battery while charging.***

### ***Understanding the M Waterproof Battery LED (light-emitting diode) Indicators***

The M Waterproof Battery has different indicator lights that are located within the M Waterproof Battery.

The LED has 3 color options:

- **Blue: M Waterproof Battery is charging.**
- **Green: M Waterproof Battery is fully charged.**
- **Red: Fault condition.\***

***\*Unplug the M Waterproof Battery and then try to charge again. If the red LED illuminates again, contact your cochlear implant professional.***

### ***Understanding the M Waterproof Battery Audible Alarm***

The M Waterproof Battery produces a constant, audible alarm when the headpiece is unlocked from the implant in order to warn caregivers when the headpiece becomes unlocked. The audible alarm is a programmable feature that can be enabled or disabled by your cochlear implant professional.

### ***Streaming Audio while using the M Waterproof Battery***

When the processor is connected to a Bluetooth-enabled device, Phonak Roger transmitter, Phonak PartnerMic, or Phonak TV Connector, you will be able to stream audio from that device directly to your sound processor when it is in the M Waterproof Battery.

The microphone in the headpiece cannot be used for hands-free phone calls via Bluetooth. If you receive a phone call when the processor is in the M Waterproof Battery, you should change the audio routing on your phone to a different source (e.g., phone or speaker phone). Please refer to the M Waterproof Battery Instructions for Use for additional information.

## ***M Wearing Clip, M Snuggie, and M Retention Cuff***

### ***M Wearing Clip***

Ensure the sound processor is set to an off-ear program utilizing the headpiece microphone when using the M Wearing Clip. Confirm that a headpiece with a microphone is attached to the sound processor. It is recommended that the M Earhook be used when the processor is worn with the M Wearing Clip.

The M Wearing Clip fits all sizes of the M Batteries and the M Zn-Air Battery Pak.

Slide the latch in order to open the sound processor enclosure.

Place the sound processor into the enclosure and close the door.

Connect the headpiece cable to the sound processor.

Squeeze the clip to attach in the desired location.

### ***M Snuggie***

The M Snuggie fits all sizes of the M Batteries and the M Zn-Air Battery Pak.

Slide the rings on to the band, with the smallest ring at one end. Align the band along the sound processor. Slide one ring around the sound processor below the multi-function button and slide one ring around the battery. Slide the earhook ring around the earhook.

Place the sound processor on the ear and wrap the band around the ear, trimming the band as need to ensure the band fits snugly and comfortably around the ear. Connect the ends of the band to form a loop.

### ***M Retention Cuff***

The M Retention Cuff fits all sizes of the M Batteries and the M Zn-Air Battery Pak.

Slide the sound processor into the plastic casing at the top of the plastic casing. As needed, spread the top arms of the plastic casing to secure the top of the sound processor and the earhook with the plastic casing.

Place the sound processor onto the ear and shape the retention wire around to the front of the ear to ensure the M Retention Cuff fits snugly and comfortably around the ear.

To remove the sound processor from the plastic casing, slide the sound process up through the top of the plastic casing by pressing on the bottom of the battery.

Please refer to the M Wearing Clip, M Snuggie, and M Retention Cuff Instructions for Use for additional information.

### ***M Listening Check***

The M Listening Check can be used to listen to the following sound sources:


- **Microphones (front and rear, headpiece, T-Mic)**
- **T-Coil**
- **Wireless input (via Bluetooth, Phonak Roger transmitters, or Phonak AirStream accessories)**

#### ***Listening to the desired input:***


Remove the battery from the sound processor (if not already removed). Hold the M Listening Check with one hand using the grip indents on the sides for traction. Using the other hand, grasp the M Listening Check over the AB logo and pull out.

Turn the AB logo portion of the M Listening Check 90 degrees.

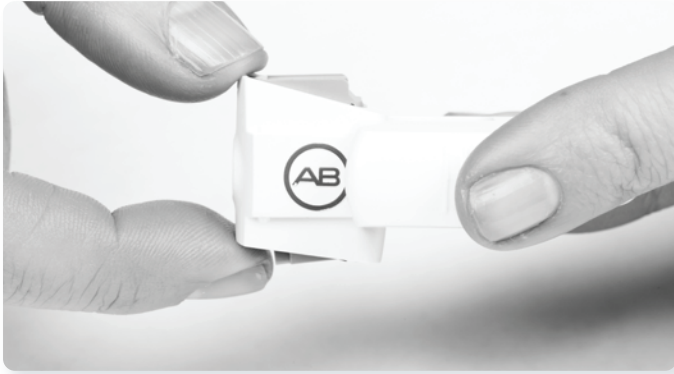Bring your hands together, sliding the AB logo portion into the portion of the M Listening Check in your other hand. You should feel the two portions snap into place. Once in place, the sound processor and M Battery connectors will be exposed.

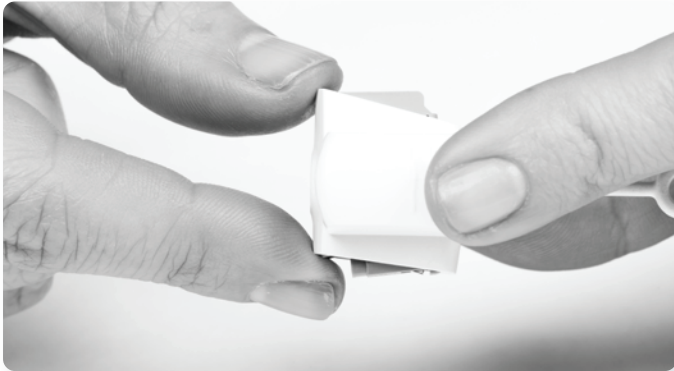Carefully align the battery connector on the sound processor with the battery connector on the M Listening check and connect the top of the M Listening Check module to the sound processor.

Carefully align the battery connector on the power source with the battery connector on the M Listening check and connect the bottom of the M Listening Check to the power source.

Connect the supplied earbuds to the 3.5mm auxiliary jack on the side of the M Listening Check.

Place earbuds in ears of the listener. When the M Listening Check is attached, the sound processor immediately enters listening check mode. You will be listening to the input sound source of the start-up program. If you would like to listen to the other sound source configurations, use the program button to cycle through the programs or begin wireless streaming from the wireless device.

If you select a T-coil program, make sure you are synced to either a loop or Roger myLink. If you begin wireless streaming from a wireless device, be cautious and turn volume down on that wireless device. If listening to a Roger transmitter or the PartnerMic, provide input to the Roger transmitter or PartnerMic to assess audio quality.


Once listening check is complete, disconnect the M Listening Check from the M Battery.

Remove the sound processor from the M Listening Check. Reconnect the M Battery to sound processor and select desired settings for use.

To close the M Listening Check when not in use, remove earbuds, and then grasp the grip indents on the side of the M Listening Check with one hand while grasping the sound processor and battery interfaces with the other hand. Carefully pull your hands apart until the two pieces disconnect.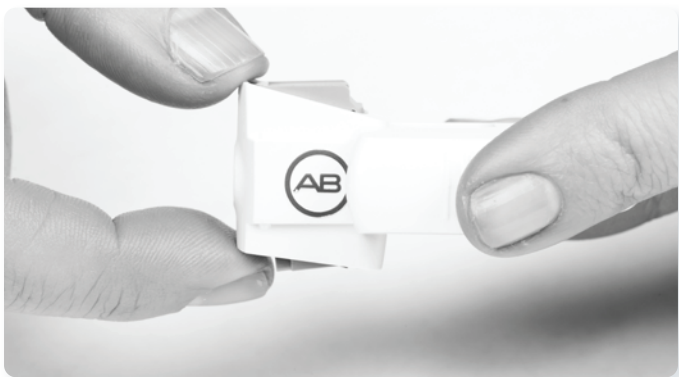

Turn one piece 90 degrees, align the AB logo piece to slide into the other piece. Push together until it snaps into place.

***Note: The M Listening Check requires a charged power source to function correctly. If the power source being used to conduct the listening check has been depleted to an inadequate charge level, the user will not be able to complete the task. If the user attaches a discharged rechargeable M Battery or a discharged M Zn-Air Battery Pak, no sound will be heard through the Listening Check Earbuds.***

Please refer to the M Listening Check Instructions for Use for additional information.

### ***AB Remote Mobile Application***

AB Remote is a smartphone app that is designed to provide user control over volume and program changes as well as provide access to help information. The AB Remote app provides direct connectivity and control of Naída CI M90 sound processors and Phonak Naída Link M hearing instruments.

Installing AB Remote is very simple. It installs just like other apps that are installed on your phone.

- **Turn on your smart phone.**
- **Open your app store app.**
- **Search for “AB Remote” or “Advanced Bionics”.**
- **Select the app and install it using your app store account.**

AB Remote is available on the Apple App Store and the Google Play store.

# Caring for Naída CI M90 and Accessories

## Storing the Naída CI M90 and its Accessories

When not in use, store the Naída CI M90 and its accessories in the equipment case provided.

## Recommended Operating and Storage Temperature and Humidity Ranges

CONDITION	MINIMUM	MAXIMUM
Operating Temperature	0°C (32°F)	45°C (115°F)
Storage Temperature	-20°C (-13°F)	55°C (131°F)
Relative Humidity	0%	95%

## Cleaning and Maintenance

Wipe the outside of the products with a soft, dry cloth. Do not immerse in liquids or use cleaning agents. If the Naída CI M90 sound processor is exposed to moisture, place the device in a drying system before next use. You should remove the devices from the ear before using hair spray or applying cosmetics because these products may cause damage.

## Ingress Protection (IP) Ratings

The following product configuration has an IP rating of 22 (protection against insertion of solid objects  $\geq 12.5$  mm diameter; protection against failure due to dripping water when tilted up to 15°):

- **The Naída CI M90 sound processor with the M Listening Check, an M Battery or M Zn-Air Battery Pak, a M T-Mic, M Earhook, or M Acoustic Earhook, and a headpiece.**

The following product configuration has an IP rating of 52 (protection against dust; protection against failure due to dripping water when tilted up to 15°):

- **The Naída CI M90 sound processor inside the M Waterproof Battery with a non-waterproof headpiece.**

The following product configurations have an IP rating of 54 (protection against dust; protection against failure due to water splashes from all directions):

- **The Naída CI M90 sound processor with an M Battery or M Zn-Air Battery Pak, an M T-Mic or M Acoustic Earhook, and a Slim HP, Slim HP Mic, or Slim HP Standard.**

- **The Naída CI M90 sound processor with an M Zn-Air Battery Pak, an M Earhook, and a Slim HP, Slim HP Mic, or Slim HP Standard.**

The following product configuration has an IP rating of 57 (protection against dust; protection against failure due to one-time immersion for 30 minutes to a depth up to 1 meter and after drying overnight in a drying system):

- **The Naída CI M90 sound processor with an M Battery, an M Earhook, and a Slim HP, Slim HP Mic, or Slim HP Standard.**

The following product configuration has an IP rating of 68 (complete protection against dust penetration; protection against failure due to continuous immersion in water up to 3 meters):

- **The Naída CI M90 sound processor inside the M Waterproof Battery with a waterproof headpiece and cable.**

## Disposing of Naída CI M90 and Accessories

Dispose of the Naída CI M90 sound processor and its accessories in accordance with applicable national and local regulations.

This instrument is certified under:

- **FCC ID: 2AU60-ABBTE2**
- **IC: 25853-ABBTE2**

*The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Sonova AG is under license.*

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Advanced Bionics LLC