Advanced Bionics AG  
Laubisrütistrasse 28  
8712 Stäfa, Switzerland  
+41 58 928 78 00

Manufactured by:  
Advanced Bionics LLC  
California, U.S.A.  
+1 661 362 1400

[AdvancedBionics.com](https://www.AdvancedBionics.com)

029-M602-85 Rev B

© 2015 Advanced Bionics AG and affiliates. All Rights Reserved.

# MRI Antenna Coil Cover

CI-7521

instructions for use

English, Français, Español, Deutsch, Nederlands, Português

## Labeling symbols and their meanings:

 Model number	 Date of manufacture
 Lot Number	 Manufacturer
 Do Not Reuse	 See Instructions for Use
	MR Conditional: Specific circumstances are required for the patient to undergo an MRI procedure with this device in place.

For detailed information regarding the use of an MRI scanner with a HiRes™ Ultra device, please refer to the **Instructions for Use HiResolution™ Bionic Ear System**, or contact Advanced Bionics Technical Support.

## MRI Warnings

*Do not allow patients with a HiRes™ cochlear implant to be in the area of an MRI scanner unless the following conditions have been met:*

- *The bandaging protocol recommended by Advanced Bionics is followed when the patient undergoes an MRI procedure with the magnet left in place or,*
- *For cases that would clinically benefit from reduced device artifact (for example some head or neck scans), the internal magnet is surgically removed and*

*possibly replaced with the Magnet Insert Dummy before the patient undergoes an MRI procedure.*

- *The external sound processor and headpiece are removed before entering a room where an MRI scanner is located.*
- *MRI preparations must be conducted outside of the MRI room.*
- *Verify that the implant, or both implants if bilaterally implanted, are compatible for conducting an MRI before proceeding. Failure to do so can lead to device movement, device damage, magnet movement, patient discomfort, or trauma and pain to the patient.*
- *MRI procedures are contraindicated for CLARION (CI and CII) Cochlear Implant recipients.*

## **Intended Use**

The MRI Antenna Coil Cover is intended to be used along with a bandaging protocol to allow the patient to undergo an MRI procedure with the magnet left in place. Instructions for use must be followed, including the recommended bandaging protocol.

# MRI Antenna Coil Cover

## Compression Dressing prior to MRI Procedure with HiRes Family of cochlear implant Users

This document will provide general guidance for applying appropriate compression dressing over the site of the implanted device (and magnet) in order to perform an MRI procedure. If the patient has bilateral cochlear implants, a single compression dressing to cover both implant sites is sufficient.

### Required Materials

- Coach Bandage  
3 inch (8 cm) width  
**Johnson and Johnson**
- Flexible tape  
Latex Free  
2 – 2.5 cm width  
**3M – Nexcare (or equivalent)**
- Patient Headpiece
- MRI Antenna Coil Cover  
CI-7521  
**Advanced Bionics**
- Marking Pen

## Procedure

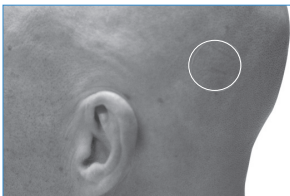
1. Place patient in sitting position to allow access to the implant site.

**NOTE:** *If the patient is bilaterally implanted, prepare to locate both implant devices using the following steps. Both implant locations must have compression dressing placed over their magnets.*


2. Obtain the patient headpiece (with cable removed) and place over the implant site. Allow magnetic attraction to occur.

3. Once in place, hold the headpiece while drawing a line around it.

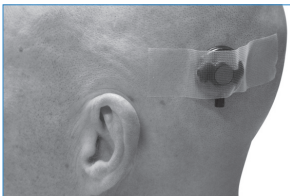4. Obtain the MRI Antenna Coil Cover and place it over the drawn location. Once centered, place flexible tape over the site to maintain the location.

5. Wrap the Coach bandage around the head without stretching.
6. Determine the head circumference by marking on the bandage.
7. Unwrap the bandage and wrap again, this time tightly, by stretching the marked line an additional half turn around the head.
8. Continue wrapping the bandage for an additional 1 and a half turns, resulting in 3 full turns total.

## After the MRI

Once the MRI procedure has been completed, gently remove the Coach Wrap, tape, and the MRI Antenna Coil Cover.

Discard all components including the MRI Antenna Coil Cover.

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