Advanced Bionics AG
Laubisrütistrasse 28
8712 Stäfa, Switzerland
+41 58 928 78 00

Manufactured by:
Advanced Bionics LLC
California, U.S.A.
+1 661 362 1400

AdvancedBionics.com

029-M678-96

029-M678-96 Rev A 2017-10-05

© 2017 Advanced Bionics AG. All Rights Reserved.

Naída CI

By Advanced Bionics

Naída CI Q90
Acoustic Earhook
CI-5850

instructions for use

English, Español, Português

Symbols Glossary

This table provides an explanation of symbols that may be found on products and packaging.

SYMBOL	SYMBOL MEANING
	Model number
	Date of manufacture
	Manufacturer
	Lot Number
	Type of Protection: B
	Suitable temperature range for transport and storage
	See Instructions for Use
	Suitable relative humidity range for transport and storage
	Suitable atmospheric pressure range for transport and storage
	Dispose of in accordance with applicable national and local regulations
	Caution (refer to "Cautions & Warnings" section for further information)
	Fragile

IP54	Ingress Protection (IP) Rating: The Naída CI Q90 Acoustic Earhook, in the expected wearing configuration, carries a rating of IP54. IP5X: Protected against access to hazardous parts. Dust protected. IPX4: Protected against water splashing from any angle.
-------------	--

Introduction

Carefully read all instructions prior to use. Observe all warnings and precautions noted throughout these instructions.

Cautions & Warnings

- **CHOKING HAZARD:** Acoustic Earhook contains small parts that could cause choking if swallowed.
- If any parts are swallowed consult a physician or hospital immediately.
- The Acoustic Earhook should only be used by the intended recipient and not by any other individual.
- Do not allow children to play with or operate the drying system unattended.
- Using your Acoustic Earhook contradictory to the intended use (i.e. mouthing, chewing) may cause bodily harm.
- Remove your sound processor and headpiece before entering a room where an MRI scanner is located.
- Remove Acoustic Earhook to stop stimulation if uncomfortably loud sounds are heard.

- If the sound processor or accessories become unusually hot or warm, discontinue use immediately and contact your cochlear implant professional.
- Do not expose the Acoustic Earhook to extreme heat, such as an oven, microwave or hair dryer.
- Do not use the Acoustic Earhook unless the Wax Guard is properly seated and the appropriate dome is attached.
- In the unlikely event that the dome gets stuck in the ear canal, it is strongly recommended to seek medical assistance to safely remove the part.
- Exposure of the receiver to high humidity, sweat, or water could damage the receiver.
- In the unlikely event that the Wax Guard becomes dislodged from the receiver while in the ear canal, contact your cochlear implant professional. Do NOT try to remove the Wax Guard from your ear canal yourself.

Intended Use


The Naída CI Q90 Acoustic Earhook (Acoustic Earhook) is intended to provide low frequency amplification to Advanced Bionics' cochlear implant recipients.

The Acoustic Earhook is indicated for Advanced Bionics cochlear implant recipients with unaided air conduction thresholds less than or equal to 90 dB HL between 125 Hz and 1600 Hz following surgery. The Acoustic Earhook should only be used when behavioral audiometric thresholds can be obtained and the recipient can provide feedback regarding sound quality. Speech perception testing should be completed before and after fitting with the acoustic component to ensure

that the recipient performs as well, if not better, with the acoustic component than without it.

Description of the Device

The Naída CI Q90 Acoustic Earhook allows for the addition of acoustic amplification to the electrical stimulation traditionally produced by a cochlear implant. The Acoustic Earhook attaches to the Naída CI Q90 Sound Processor with the **processor connector**. From the processor connector, a **tube** extends to the **xReceiver Unit** that transfers sound inputs through the **sound outlet** into the ear's auditory canal. The Acoustic Earhook contains a **dash** on the side of the xReceiver Unit, that acts as a guide for the **loop** of the optional Retention Hook, and a colored **indicator** on the xReceiver Unit, blue for the left ear and red for the right ear.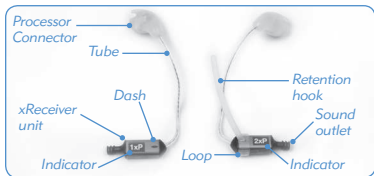

Operating Information

The Naída CI Q90 Acoustic Earhook is compatible with the Naída CI Q90 Sound Processor only, and SoundWave software, version 3.1 and later. Please ensure that the Naída CI Q90 Sound Processor has been programmed


properly to delivery acoustic amplification through the Acoustic Earhook before first use.

Set-up Instructions

The Acoustic Earhook has been provided to you partially assembled. Before using the Acoustic Earhook you **must** ensure:

1. A Wax Guard (e.g. Cerustop) is properly attached to the Sound Outlet.
2. A Closed Dome or Power Dome is attached to the Sound Outlet over top of the Wax Guard.
3. The Acoustic Earhook is connected to your Naída CI Q90 Sound Processor.
4. Your cochlear implant professional has programmed your Naída CI Q90 Sound Processor for use with your Naída CI Q90 Acoustic Earhook.

WAX GUARD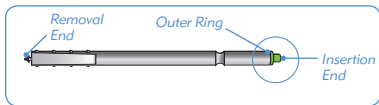

When using the Wax Guard, it is important that you:

- Do not pry or twist the Wax Guard.
- Do not force the Wax Guard into the Sound Outlet.
- Never reuse a Wax Guard.

- Remove and insert another Wax Guard if it fits loosely.
- Discard the Wax Guard after use.

Removing & Inserting Wax Guard on the xReceiver Unit


Use the empty Removal End of the Wax Guard to remove the used Wax Guard from the Sound Outlet on the xReceiver Unit by:

- Inserting the Removal End into the Sound Outlet until the rim of the Wax Guard is touching the rim of the Sound Outlet.
- Then slowly pulling the Wax Guard straight out.

Use the Insertion End of the Wax Guard to insert the new Wax Guard onto the Sound Outlet on the xReceiver Unit by:

- Pressing the Insertion End straight into the opening of the Sound Outlet until the Outer Ring of the Wax Guard sits flat on the Sound Outlet.
- Then pulling the Wax Guard straight out.

CLOSED DOME or POWER DOME

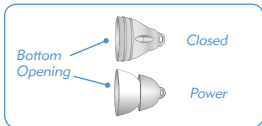Attaching and Removing Closed Dome or Power Dome on xReceiver Unit

The Closed Dome or Power Dome should only be attached to the Sound Outlet on the xReceiver Unit if a Wax Guard is properly inserted on the Sound Outlet.

Attach the dome to the Sound Outlet on the xReceiver Unit by:

- Gently grasping the xReceiver Unit with one hand
- With the other hand, aligning the bottom opening of the dome with the Sound Outlet of the xReceiver Unit
- Gently pressing the dome into place until it is securely attached.

Remove the dome from the Sound Outlet on the xReceiver Unit by:

- Gently grasping the xReceiver Unit with one hand
- With the other hand, grasping the dome
- Pulling it away from the xReceiver Unit until disconnected from the Sound Outlet of the xReceiver Unit.

For a more custom fit, the Phonak SlimTip is an optional earpiece that may be used in place of the provided Closed Dome or Power Dome. The SlimTip must be ordered from Phonak; it is not provided with the Naída CI Q90 Acoustic Earhook. Contact your cochlear implant professional for more details.

Connecting & Removing the Acoustic Earhook

Connecting the Naída CI Q90 Acoustic Earhook

- Remove the power source from the Naída CI Q90 Sound Processor.
- Remove the standard Earhook already attached to the Naída CI Q90 Sound Processor.
- Align the Naída CI Q90 Acoustic Earhook flush with the Naída CI Q90 Sound Processor.
- Hold the Acoustic Earhook firmly at the top and push towards the Sound Processor until it meets flush with the processor.
- Using the provided T-Mic 2 / Earhook tool, place one of the pins in the small hole on the side of the tool. This will help to steady the pin as you line it up with the pin hole on the Acoustic Earhook.
- Line up the pin with the hole in the Acoustic Earhook and gently push it through the hole. The pin will move from one side of the Sound Processor, through the Acoustic Earhook, to the other side of the Sound Processor, holding the Acoustic Earhook in place. The pin should not come out of the other side.
- If a small portion of the pin extends beyond the Sound Processor, use the side of the tool to push the pin inwards so its lies flush with the Sound Processor.
- Pull the tool away from the processor, leaving the pin in place.

Removing the Naída CI Q90 Acoustic Earhook

- Remove the power source from the Naída CI Q90 Sound Processor.

- Align the pointed end of the T-Mic 2 / Earhook removal tool with the hole on the processor that contains the pin holding the Naída CI Q90 Acoustic Earhook on the Naída CI Q90 Sound Processor.
- Gently press the tool in the hole, pushing the pin out the other side.
- The pin does not need to be removed from the Naída CI Q90 Sound Processor. As long as the tool is pushed in flush on one side of the Naída CI Q90 Sound Processor, the pin should be displaced enough to adequately remove the Acoustic Earhook.
- Remove the tool from the Naída CI Q90 Sound Processor.
- Gently grasp the Acoustic Earhook at the top near where it connects with the Naída CI Q90 Sound Processor and pull forward until it is separated from the Naída CI Q90 Sound Processor.

Retention Hook

Attaching the Retention Hook to the xReceiver Unit

- Grasp the xReceiver Unit in one hand.
- With the other hand, align the retention hook so that the loop is in front of the xReceiver Unit, and the retention hook points forward, away from the Sound Processor.
- Gently slide the loop of the retention hook into the xReceiver Unit until the retention hook covers the dash.

Removing the Retention Hook from the xReceiver Unit

- Grasp the xReceiver Unit in one hand.
- With the other hand, grasp the loop of the retention hook that surrounds the xReceiver Unit and carefully slide the retention hook away from the tube of the Acoustic Earhook and off the xReceiver Unit.

It may be necessary to use your nails to get a good grasp and remove the retention hook.

Do not hold on to or pull the tube while removing the retention hook.

Operating Instructions

Inserting the xReceiver Unit into the Ear Canal

- Place the Naída CI Q90 Sound Processor behind the appropriate ear.
- Place the Naída CI Q90 Acoustic Earhook tube between the thumb and forefinger.
- Insert the Dome deep enough into the ear canal so that the tube lies flat on the ear.
- If utilizing a retention hook, press it into the bowl of the ear in order to retain the Dome in the ear canal.

If you have problems inserting the Dome into the ear canal:

- Use your other hand to gently pull your earlobe downwards, as this opens the ear canal a little more.
- Then gradually rotate the Dome until it fits correctly.

Removing the xReceiver Unit from the Ear Canal

- Simply grasp the Acoustic Earhook tube and carefully remove it from your ear.

Operating & Storage Instructions

Recommended Operating Conditions

Temperature: 0°C (32°F) - 45 °C (113 °F)

Relative Humidity: up to 90% RH

Atmospheric Pressure: 70 kPa – 106 kPa

Recommended Storage Conditions

Temperature: -20°C (-4°F) - 55 °C (131 °F)

Relative Humidity: up to 90% RH

Cleaning & Maintenance

For more in depth maintenance instructions or for more than basic cleaning, please contact your cochlear implant professional.

- Your Acoustic Earhook should be inspected daily for earwax and moisture deposits.
- If the Acoustic Earhook is exposed to moisture or humidity, it is best to dry it daily.
- If utilizing a Dome, it should be replaced every four weeks, or sooner if damage is noted.
- Replace the Wax Guard on the xReceiver Unit every four to eight weeks, or sooner if it becomes visibly dirty, blocked, or the sound seems quieter or lower.

- Remove the Acoustic Earhook before using hair spray or applying cosmetics, as these products may damage the Acoustic Earhook.

To clean your Acoustic Earhook:

1. Remove the Dome from the xReceiver Unit
2. Using a lint-free cloth or moist cleansing wipe, gently clean the Dome and xReceiver Unit. **Never use cleaning agents such as household detergents, soap, etc. to clean any parts of your Acoustic Earhook.**
3. Prior to reattaching the Dome to the xReceiver Unit, ensure that all parts are completely dry.

User Assistance Information

In the event that you experience any issues with your product, please contact your cochlear implant professional or the manufacturer. Do not attempt to service or modify your product as doing so may compromise system performance and will void the manufacturer's warranty. Products should be serviced only at Advanced Bionics.

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