ADVANCED  
BIONICS

POWERFUL CONNECTIONS

## **M WATERPROOF BATTERY and CLIP**

CI-5068, CI-7323

*instructions for use*

*English, Français, Español, Deutsch, Nederlands, Italiano*

# Contents

<b>PURPOSE AND SCOPE OF IFU</b>	<b>5</b>
<b>LIMITATIONS AND CONTRAINDICATIONS</b>	<b>5</b>
<b>PRECAUTIONS, CAUTIONS, AND WARNINGS</b>	<b>5</b>
<b>UNDESIRABLE SIDE EFFECTS</b>	<b>10</b>
<b>PRODUCT DESCRIPTION</b>	<b>10</b>
<b>INDICATIONS FOR USE</b>	<b>12</b>
<b>INTENDED PURPOSE</b>	<b>13</b>
<i>Intended Users</i>	14
<i>Intended Use Environment</i>	14
<b>COMPATIBILITY</b>	<b>14</b>
<b>OPERATING INSTRUCTIONS</b>	<b>15</b>
<i>General Information for Using the M Waterproof Battery</i>	15
<i>Inserting the Sound Processor in the M Waterproof Battery</i>	16
<i>Removing the Sound Processor from the M Waterproof Battery</i>	20
<i>Replacing the O-Ring on the M Waterproof Battery</i>	21
<i>Charging the M Waterproof Battery</i>	23
<i>Understanding the M Waterproof Battery LED (light-emitting diode) Indicators</i>	24
<i>Understanding the M Waterproof Battery Audible Alarm</i>	24
<i>Streaming Audio while using the M Waterproof Battery</i>	25

<b>CARE AND MAINTENANCE</b>	<b>25</b>
<i>Storing the M Waterproof Battery</i>	25
<i>Recommended Operating, Storage Temperature,     and Humidity Ranges</i>	25
<i>Cleaning and Maintenance</i>	26
<i>Expected Life</i>	26
<i>Ingress Protection (IP) Ratings</i>	26
<b>DISPOSAL AND HAZARDOUS MATERIALS</b>	<b>27</b>
<b>AVAILABLE PRODUCT OPTIONS AND MODEL NUMBERS</b>	<b>27</b>
<b>CLINICAL BENEFITS</b>	<b>28</b>

## SYMBOLS TABLE

### *Labeling symbols and their meanings*

	European Community Mark of Conformity
	Date of Manufacture
	Manufacturer
	Model number
	Serial number
	LOT number
	European authorized representative
	Unique Device Identifier
	Medical Device
	Type of Protection: BF
	Suitable temperature range for transport and storage

	See Instructions for Use
	Fragile
	Suitable relative humidity range for transport and storage
	Caution (refer to “Cautions & Warnings” section for further information)
	Dispose of in accordance with applicable national and local regulations
	MR Unsafe
<b>IPxx</b>	Ingress Protection Rating

## PURPOSE AND SCOPE OF IFU

This IFU is designed to help cochlear implant recipients, and their caregivers if applicable, to understand the use and care of the M Waterproof Battery. It is also meant to be used by hearing care professional trained in the fitting of cochlear implants to counsel the recipients on the use of the M Waterproof Battery and to troubleshoot problems.

## Limitations and Contraindications

As the M Waterproof Battery is a component of the HiResolution™ Bionic Ear System, the following contraindications stated for the HiResolution Bionic Ear System are applicable: Deafness due to lesions of the acoustic nerve or central auditory pathway; cochlear ossification that prevents electrode insertion; absence of cochlear development.

## Precautions, Cautions, and Warnings

- *This device should be used only by the individual for whom it is prescribed.*
- *It is recommended that you charge the M Waterproof Battery fully prior to first use, as the M Waterproof Battery shipped from Advanced Bionics is not fully charged.*
- *The M Waterproof Battery comes with a power supply designed for the specifications of the M Waterproof Battery. Do not use any other power supply unless it is supplied by Advanced Bionics. If needed, call Advanced Bionics for a power supply replacement.*
- *The M Waterproof Battery and power supply should be operated in an open area to ensure adequate airflow.*

*While no injury cases have resulted, components may become hot during normal use or a fault condition. If the device's temperature results in discomfort or pain when touched, disconnect the power source and contact your cochlear implant professional.*

- The M Waterproof Battery may reach a temperature of 48°C (119°F) while charging. Do not wear the M Waterproof Battery while charging.*
- CHOKING HAZARD: Contains small parts that pose a hazard of inhalation, choking, or ingestion. If any parts are swallowed or inhaled, consult a physician or hospital immediately.*
- Do not leave children unattended with or allow children to play with the M Waterproof Battery.*
- Do not allow children to place the M Waterproof Battery in the mouth.*
- Remove your sound processor and consult your cochlear implant professional if uncomfortable sounds are heard or in case of discomfort, pain or skin irritation.*
- Do not dispose of the M Waterproof Battery in fire, as it may explode.*
- To prevent injury or burns, do not allow metal objects, such as keys or coins, to contact or short circuit the battery terminals. The M Waterproof Battery may present risk of fire or chemical burn if mistreated. Do not disassemble. Do not heat above 100°C/212°F or incinerate.*
- Remove your sound processor from the M Waterproof Battery when the battery is discharged to prevent damage from possible leaking.*

- *If the M Waterproof Battery shows visible signs of leakage, dispose of it in accordance with local regulations, as detailed in the “Disposal” section of this document.*
- *Do not allow leaking battery fluid to come into contact with skin, mouth, or eyes.*
- *Do not expose the M Waterproof Battery to heat (e.g., do not store in direct sunlight or in a hot car).*
- *Carry the M Waterproof Battery in the carrying case provided or in a sealed plastic bag. The M Waterproof Battery can short circuit if it comes into contact with metallic objects such as coins or keys.*
- *High levels of static electricity may damage electronic components of your sound processor or implant. Care should be taken to avoid exposure of the system to static electricity.*
- *The M Waterproof Battery O-Ring should be inspected prior to each use and will need to be changed periodically. If damage to the O-Ring is observed or suspected, replace prior to resuming use. Do not continue use of the M Waterproof Battery if it is damaged. Doing so may void the warranty and may compromise system performance. If used improperly or damaged, you may be liable for the repair cost of the product. This includes, but is not limited to, water or particulate matter damage.*
- *When the M Waterproof Battery is moved from a hot environment (e.g., a sauna or whirlpool) to a colder environment (e.g., a pool), condensation may form inside the case. If condensation occurs, remove the processor and place in a drying system overnight to ensure proper*

*drying of the sound processor. Wipe the battery case with a soft, lint-free cloth and air dry overnight.*

- If water or particle ingress is observed, cease use of the M Waterproof Battery immediately, remove the processor from the enclosure and place in a drying system overnight to ensure proper drying of the sound processor. Wipe the battery case with a soft, lint-free cloth and air dry overnight.*
- Following exposure to particulate matter, such as sand or dirt, clean the outside of the case with water and a dry cloth prior to opening.*
- The Waterproof Battery and O-Ring should be rinsed and dried following submersion in salt or chlorine treated water.*
- Do NOT store the M Waterproof Battery in direct sunlight or inside a vehicle during extremely hot or cold weather. The O-Ring should also not be left in direct sunlight. To prevent excessive sun exposure, cover the M Waterproof Battery and O-Ring with a towel.*
- When sunbathing or after water use, please remove the sound processor from the M Waterproof Battery or cover the M Waterproof Battery and sound processor to shade it from direct sunlight and prevent excessive temperatures.*
- Do not open or close the M Waterproof Battery while in a water environment.*

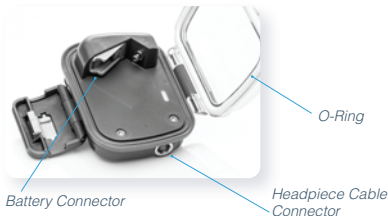
- *Cables are not intended to be removed or replaced while the M Waterproof Battery is submerged. If the cable accidentally detaches during water use, remove the M Waterproof Battery from the water and shake excess water from the case prior to reconnecting the cable. Wipe the battery case with a soft, lint-free cloth and air dry overnight.*
- *To prevent damage to accessories compatible with the sound processor (e.g., Listening Check, Phonak Roger devices, Phonak RemoteControl), it is important to cease use prior to engaging in activities in or near water. Once in a dry environment, the recipient may resume use of the accessories with the sound processor in the M Waterproof Battery.*
- *To avoid an increased risk of strangulation, do not use cables longer than 81cm (32 inches) for young children.*
- *Although electromagnetic emissions of the M Waterproof Battery have been confirmed within safe limits, some other devices could be sensitive to such emissions. If you notice other devices do not behave as expected when your M Waterproof Battery is nearby, separate the devices.*
- *In the event that you experience any issues with your product, please contact your cochlear implant professional or the manufacturer. Do not attempt to service or modify the Naída CI M, Sky CI M, or its accessories. Doing so may compromise system performance and will void the manufacturer's warranty. Products should be serviced only at Advanced Bionics and damaged products should be returned to Advanced Bionics.*

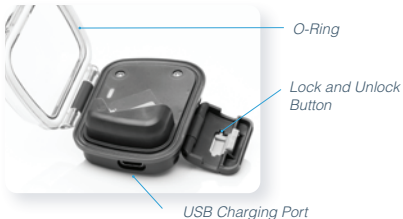
## Undesirable Side Effects

Undesirable side effects of your Naída CI M or Sky CI M system may include skin irritation and discomfort, device overheating, or overly loud sounds.

## Product Description


The M Waterproof Battery is a Lithium Ion rechargeable battery. It features an enclosure for your Naída™ CI M or Sky CI™ M sound processor to protect the sound processor from moisture and debris.

PRODUCT	BATTERY CAPACITY (MAH)
M Waterproof Battery	225

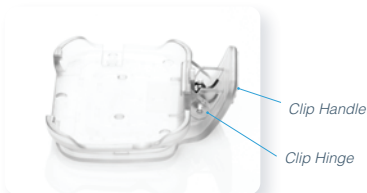The M Waterproof Battery can be worn in different locations using the M Waterproof Battery Clip.

## INDICATIONS FOR USE

The M Waterproof Battery is an external component of the HiResolution™ Bionic Ear System, which is intended to restore a level of auditory sensation to individuals with severe-to-profound sensorineural hearing loss via electrical stimulation of the auditory nerve.

### Adults

- *18 years of age or older.*
- *Severe-to-profound bilateral sensorineural hearing loss or severe-to-profound unilateral hearing loss.*
- *Post-lingual onset of severe or profound hearing loss.*
- *Limited benefit from appropriately fitted hearing aids, defined as scoring 50% or less on a test of open-set sentence recognition (HINT Sentences).*

### Children

- *12 months through 17 years of age.*
- *Severe-to-profound bilateral sensorineural deafness or severe-to-profound unilateral hearing loss.*
- *Use of appropriately fitted hearing aids for at least 6 months in children 2 through 17 years of age, or at least 3 months in children 12 through 23 months of age. The minimum duration of hearing aid use is waived if x-rays indicate ossification of the cochlea.*
- *Little or no benefit from appropriately fitted hearing aids. In younger children (< 4 years of age), lack of benefit is defined as a failure to reach developmentally appropriate auditory milestones (such as spontaneous response to name in quiet or to environmental sounds)*

*measured using the Infant-Toddler Meaningful Auditory Integration Scale or Meaningful Auditory Integration Scale or  $\leq 20\%$  correct on a simple open-set word recognition test (Multisyllabic Lexical Neighborhood Test) administered using monitored live voice (70 dB SPL). In older children ( $\geq 4$  years of age), lack of hearing aid benefit is defined as scoring  $\leq 12\%$  on a difficult open-set word recognition test (Phonetically Balanced-Kindergarten Test) or  $\leq 30\%$  on an open-set sentence test (Hearing In Noise Test for Children) administered using recorded materials in the sound field (70 dB SPL).*

## **Intended Purpose**

The M Waterproof Battery is an accessory of an auditory active-implantable system, the HiResolution Bionic Ear system. The HiResolution Bionic Ear system is intended to provide auditory sensation via electrical stimulation of the auditory nerve for individuals with severe to profound bilateral or unilateral sensorineural hearing loss. Severe hearing loss is defined as audiometric thresholds greater than or equal to 70 dB HL, but less than 90 dB HL. Profound hearing loss is defined as audiometric thresholds greater than or equal to 90 dB HL. The external components work together with the implant to bypass the damaged part of the inner ear and convert sound picked up by the microphone or streamed via wireless communication into electrical signals that are used by the cochlear implant to enable hearing.

The M Waterproof Battery is intended to power the Advanced Bionics Naída CI M and Sky CI M Sound Processors and protect the sound processor for moisture and debris. Connecting the sound processor to the

connector within the M Waterproof Battery will power on the sound processor. Removing the sound processor from the M Waterproof Battery will power off the sound processor.

### ***Intended Users***

The intended users of the M Waterproof Battery are users of the Naída CI M or Sky CI M Sound Processors, their caregivers if applicable, and hearing care professionals

The recipient, or their caregiver, should, at a minimum, be capable of changing the battery, connecting the earhook, placing and removing the sound processor from their ear, and placing and removing the headpiece from the implant site.

The hearing care professional should be trained in the use and fitting of the cochlear implant system.

### ***Intended Use Environment***

The intended use environments of the M Waterproof Battery are healthcare and daily living environments, including water environments with bath water, salt water, and fresh water. The M Waterproof Battery is designed to be worn daily during the user's normal waking hours.

### **Compatibility**

The M Waterproof Battery is compatible with Naída CI M and Sky CI M Sound Processors and the Slim HP AquaMic™ headpiece, the AquaMic, AquaMic 3D, and AquaCables from Advanced Bionics.

## Operating Instructions

### ***General Information for Using the M Waterproof Battery***

- Using the supplied power adapter, it takes approximately 3 to 4 hours to fully recharge a depleted M Waterproof Battery. If the M Waterproof Battery is stored for prolonged periods of time, it may take approximately 5 to 6 hours to fully recharge the battery.
- The M Waterproof Battery does not need to be fully depleted before charging. A partially charged M Waterproof Battery may be fully charged in a shorter time period.
- You can discontinue charging the M Waterproof Battery at any time.
- You will not damage the M Waterproof Battery if it remains charging longer than the required charging time or overnight.
- It is recommended to use and recharge the M Waterproof Battery at least once every 3 months. Please note that the M Waterproof Battery will decrease in capacity with age, even if not in use. This is normal behavior for all rechargeable batteries and should not be considered a defect.
- Ensure that the selected program supports off-ear listening (headpiece microphone input). If the program does not support headpiece microphone input, select the appropriate program that does. If you are unsure, contact your cochlear implant professional for programming assistance.

- Please note that you may notice a slight sound quality difference when switching between your BTE microphone program and your headpiece microphone program.
- Do not open or close the M Waterproof Battery while in a water environment.
- Before placing the Naída CI M or Sky CI M in the M Waterproof Battery, visually inspect the O-Ring seal to make sure it is placed correctly. Confirm that the O-Ring is not dry, brittle or dirty. If it is, replace O-Ring prior to use. If damage to the O-Ring is observed or suspected, replace prior to resuming use.

### ***Inserting the Sound Processor in the M Waterproof Battery***

The M Waterproof Battery has a lock to make sure the case is not accidentally opened, preventing moisture from entering the case and damaging the sound processor. The lock is located on the front of the M Waterproof Battery.

While holding the M Waterproof Battery in one hand, use your forefinger on the opposite hand to push the lock button while using your thumb to pull the lock away from the case.

The M Earhook, M T-Mic™ microphone, or M Acoustic Earhook, battery, cable, and headpiece must be removed from the sound processor before inserting into the case, however the earhook pin can remain in the processor. Carefully align the battery connector on the sound processor with the battery connector in the M Waterproof Battery and slide the sound processor onto the battery connector within the M Waterproof Battery, connecting the cable connector with the M Waterproof battery to the sound processor.

Slide the sound processor into place until the M Waterproof Battery and headpiece cable connector are coupled.

Once the sound processor is in place, close the lid of the M Waterproof Battery. Return the latch to the closed position, confirming that the latch has clicked into place.

Connect the AquaMic headpiece cable to the M Waterproof Battery.

To attach the clip to the M Waterproof Battery, insert the hinge side of the M Waterproof Battery into the clip and press down until the M Waterproof Battery snaps into place.

To remove the clip, pull the handle of the clip back and away from the M Waterproof Battery.

### ***Removing the Sound Processor from the M Waterproof Battery***

Towel dry the M Waterproof Battery before removing the sound processor to prevent water from accidentally getting on the sound processor. Be careful not to allow water from your hair or body to fall in the M Waterproof Battery or on the sound processor.

While holding the M Waterproof Battery in one hand, use your forefinger on the opposite hand to push the lock button while using your thumb to pull the lock away from the case.

To remove the sound processor, carefully slide the processor forward, making sure not to pull up until the sound processor is free from the connectors.

Remember to wipe the battery case with a soft, lint-free cloth and air dry overnight after any water related activities. Continue use of the Slim HP or RF cable and appropriate UHP.

### ***Replacing the O-Ring on the M Waterproof Battery***

The M Waterproof Battery O-Ring should be inspected prior to each use and will need to be changed periodically. If damage to the O-Ring is observed or suspected, replace prior to resuming use. The M Waterproof Battery and O-Ring should be rinsed following submersion in salt or chlorine treated water.

To remove the existing O-Ring, use your fingers to lift a portion of the old O-Ring out of the M Waterproof Battery. Once a portion of the O-Ring is lifted out, hold the O-Ring and pull the remaining portion out of the M Waterproof Battery mounting groove.

To insert the new O-Ring, place a portion of the new O-Ring in the M Waterproof Battery mounting groove on the case. Take care to match the shape of the O-Ring correctly with the shape of the M Waterproof Battery. While holding the O-Ring in the groove with one hand, carefully stretch the remaining portion of the O-Ring into the remaining groove.

Run your thumb over the O-Ring to make sure of proper insertion. Extreme care should be taken to make sure the O-Ring is level and secure in the mounting groove.

## ***Charging the M Waterproof Battery***

### ***To use a wall outlet for charging:***

Place the M Waterproof Battery on a flat surface.

Connect the micro USB end of the power supply cable into the M Waterproof Battery.

Connect the power supply cable to the USB wall adapter.

Plug the wall adapter into the wall outlet.

### ***To use a USB port for charging:***

Place the M Waterproof Battery on a flat surface.

Connect the micro USB end of the power supply cable into the M Waterproof Battery.

Insert the USB jack into a USB port (on a computer, in a car, etc.).

***Please Note: After use, wipe the M Waterproof Battery with a dry cloth to remove moisture or debris before charging.***

*Please Note: You may charge the M Waterproof Battery while the processor is connected, but the sound processor will be powered off during charging. Do not wear the M Waterproof Battery while charging.*

### ***Understanding the M Waterproof Battery LED (light-emitting diode) Indicators***

The M Waterproof Battery has different indicator lights that are located within the M Waterproof Battery.

The LED has 3 color options:

- **Blue:** M Waterproof Battery is charging.
- **Green:** M Waterproof Battery is fully charged.
- **Red:** Fault condition.\*

*\*Unplug the M Waterproof Battery and then try to charge again. If the red LED illuminates again, contact your cochlear implant professional.*

### ***Understanding the M Waterproof Battery Audible Alarm***

The M Waterproof Battery produces a constant, audible alarm when the headpiece is unlocked from the implant in order to warn caregivers when the headpiece becomes unlocked. The audible alarm is a programmable feature that can be enabled or disabled by your cochlear implant professional.

## ***Streaming Audio while using the M Waterproof Battery***

When the processor is connected to a Bluetooth-enabled device, Phonak Roger transmitter, Phonak PartnerMic, or Phonak TV Connector, you will be able to stream audio from that device directly to your sound processor when it is in the M Waterproof Battery.

The microphone in the headpiece cannot be used for hands-free phone calls via Bluetooth. If you receive a phone call when the processor is in the M Waterproof Battery, you should change the audio routing on your phone to a different source (e.g., phone or speaker phone).

## **CARE AND MAINTENANCE**

### ***Storing the M Waterproof Battery***

When not in use, store the M Waterproof Battery in the equipment case provided.

### ***Recommended Operating, Storage Temperature, and Humidity Ranges***

CONDITION	MINIMUM	MAXIMUM
Operating Temperature	0°C (32°F)	45°C (115°F)
Storage Temperature	-20°C (-13°F)	55°C (131°F)
Relative Humidity	0%	95%

## ***Cleaning and Maintenance***

Remove the O-Ring prior to cleaning. Clean the M Waterproof Battery with a damp cloth. Do not use solvents. After exposure to moisture, wipe the battery case with a soft, lint-free cloth and air dry overnight.

Inspect the O-Ring prior to each use, as it will need to be changed periodically. If damage to the O-Ring is observed or suspected, replace prior to resuming use.

If the M Waterproof Battery is dropped, inspect it for evidence of damage or cracking. If any evidence of damage is seen, the M Waterproof Battery should be replaced.

## ***Expected Life***

The expected life of the M Waterproof Battery is 1 year.

## ***Ingress Protection (IP) Ratings***

The following product configuration has an IP rating of 68 (complete protection against dust penetration; protection against failure due to continuous immersion in water up to 3 meters):

- **The Naída CI M90 sound processor inside the M Waterproof Battery with a waterproof headpiece and cable.**

The following product configuration has an IP rating of 52 (protection against dust; protection against failure due to dripping water when tilted up to 15°):

- **The Naída CI M90 sound processor inside the M Waterproof Battery with a non-waterproof headpiece.**

## DISPOSAL AND HAZARDOUS MATERIALS

For proper disposal of rechargeable batteries in Europe, please return depleted M Waterproof Batteries to the nearest AB location or the designated programming/follow-up center. Hazardous materials: not applicable to this product.

In the United States and Canada, please call the Rechargeable Battery Recycling Corporation hotline at 877.273.2925 or visit their website at [www.call2recycle.org](http://www.call2recycle.org).

## AVAILABLE PRODUCT OPTIONS AND MODEL NUMBERS

PRODUCT OPTION	MODEL NUMBER
M Waterproof Battery	CI-5068
M Waterproof Battery Clip	CI-7323
M Waterproof Battery O-Ring	CI-7324

## CLINICAL BENEFITS

***The Intended Clinical Benefit of the M Waterproof Battery as part of the HiResolution Bionic Ear System is to:***

- *provide useful hearing to individuals with severe-to-profound hearing loss via electrical stimulation of the auditory nerve.*
- *provide a combined type of stimulation, electric plus acoustic, when sound processors are enabled with the Acoustic Earhook.*

***Users of the M Waterproof Battery in the European Union should report any serious incident to their local competent authority and to Advanced Bionics.***

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©2021 Advanced Bionics LLC. All Rights Reserved.
Advanced Bionics LLC