ADVANCED  
BIONICS

POWERFUL CONNECTIONS

## M Batteries and M Charger

CI-5551, CI-5552, CI-5553, CI-5555, CI-5607  
(Small, Medium, Large, Standard, Charger)

*instructions for use*

*English, Français, Español, Deutsch, Nederlands, Italiano*

# Contents

<b>PURPOSE AND SCOPE OF IFU</b>	<b>5</b>
<b>LIMITATIONS AND CONTRAINDICATIONS</b>	<b>5</b>
<b>PRECAUTIONS, CAUTIONS, AND WARNINGS</b>	<b>5</b>
<b>UNDESIRABLE SIDE EFFECTS</b>	<b>8</b>
<b>PRODUCT DESCRIPTION AND PERFORMANCE CHARACTERISTICS</b>	<b>8</b>
<i>Battery Capacity</i>	9
<i>Charger Information</i>	10
<b>INDICATIONS FOR USE</b>	<b>10</b>
<b>INTENDED USE</b>	<b>11</b>
<i>Intended Users</i>	12
<i>Intended Use Environment</i>	12
<b>COMPATIBILITY</b>	<b>12</b>
<b>OPERATING INSTRUCTIONS</b>	<b>13</b>
<i>General Charging Information for M Batteries</i>	13
<i>Setting up the charger</i>	14
<i>Understanding M Battery Charger LED (light-emitting diode) indicators</i>	14
<i>Inserting the M Batteries into the charger</i>	15
<i>Removing the M Batteries from the charger</i>	16
<i>Using the M Batteries with the Naída CI M and Sky CI M Sound Processors</i>	17

<b>CARE AND MAINTENANCE</b>	<b>19</b>
<i>Storing the M Batteries and M Battery Charger</i>	19
<i>Cleaning and Maintenance</i>	19
<i>Expected Life</i>	20
<i>Ingress Protection (IP) Ratings</i>	20
<b>DISPOSAL AND HAZARDOUS MATERIALS</b>	<b>21</b>
<b>AVAILABLE PRODUCT OPTIONS AND MODEL NUMBERS</b>	<b>21</b>
<i>M Battery Charger</i>	21
<i>M Battery Small</i>	21
<i>M Battery Medium</i>	22
<i>M Battery Large</i>	23
<i>M Battery Standard</i>	24
<b>CLINICAL BENEFITS</b>	<b>24</b>

## SYMBOLS TABLE

### *Labeling symbols and their meanings*

	European Community Mark of Conformity.
	Date of Manufacture
	Manufacturer
	Model number
	LOT number
	European authorized representative
	Unique Device Identifier
	Medical Device
	Type of Protection: BF
	Suitable temperature range for transport and storage
	See Instructions for Use
	Fragile

	Do not get wet
	Suitable relative humidity range for transport and storage
	Caution (refer to "Cautions & Warnings" section for further information)
	Dispose of in accordance with applicable national and local regulations
	MR Unsafe

## PURPOSE AND SCOPE OF IFU

This IFU is designed to help cochlear implant recipients, and their caregivers if applicable to understand the use and care of the M Batteries and M Battery Charger. It is also meant to be used by hearing care professionals trained in the fitting of cochlear implants to counsel the recipients on the use of the M Batteries and M Battery Charger and to troubleshoot problems.

## Limitations and Contraindications

As the M Batteries and M Battery Charger are components of the HiResolution™ Bionic Ear System, the following contraindications stated for the HiResolution Bionic Ear System are applicable: Deafness due to lesions of the acoustic nerve or central auditory pathway; cochlear ossification that prevents electrode insertion; absence of cochlear development.

## Precautions, Cautions, and Warnings

- *This device should be used only by the individual for whom it is prescribed.*
- *It is recommended that you charge the M Batteries fully prior to first use, as M Batteries shipped from Advanced Bionics are not fully charged.*
- *The M Battery Charger comes with a power supply designed for the specifications of the M Battery Charger. Do not use any other power supply unless it is supplied by Advanced Bionics. If needed, call Advanced Bionics for a power supply replacement.*

- *The M Battery Charger and power supply should be operated in an open area to ensure adequate airflow. While no injury cases have resulted, components may become hot during normal use or a fault condition. If the device's temperature results in discomfort or pain when touched, disconnect the power source and contact your cochlear implant professional.*
- *CHOKING HAZARD: Contains small parts that pose a hazard of inhalation, choking, or ingestion. If any parts are swallowed or inhaled, consult a physician or hospital immediately.*
- *Do not leave children unattended with or allow children to play with the M Battery Charger or M Battery.*
- *Do not allow children to place the M Battery Charger or M Battery in the mouth.*
- *Remove your sound processor and consult your cochlear implant professional if uncomfortable sounds are heard or in case of discomfort, pain or skin irritation.*
- *Do not dispose of M Batteries in fire, as they may explode.*
- *To prevent injury or burns, do not allow metal objects, such as keys or coins, to contact or short circuit the battery terminals. M Batteries may present risk of fire or chemical burn if mistreated. Do not disassemble. Do not heat above 100°C/212°F or incinerate.*
- *Remove M Batteries from your sound processor when they are discharged to prevent damage from possible leaking.*

- *If an M Battery shows visible signs of leakage, dispose of it in accordance with local regulations, as detailed in the "Disposal" section of this document.*
- *Do not allow leaking battery fluid to come into contact with skin, mouth, or eyes.*
- *Do not expose M Batteries to heat (e.g., do not store in direct sunlight or in a hot car).*
- *Carry M Batteries in the carrying case provided or in a sealed plastic bag. M Batteries can short circuit if they come into contact with metallic objects such as coins or keys.*
- *Do not immerse the M Batteries or M Battery Charger directly in water.*
- *High levels of static electricity may damage electronic components of your sound processor or implant. Care should be taken to avoid exposure of the system to static electricity.*
- *Only charge M Batteries using the charger(s) recommended by the manufacturer.*
- *Do not recharge disposable batteries.*
- *Although electromagnetic emissions of the M Battery Charger have been confirmed within safe limits, some other devices could be sensitive to such emissions. If you notice other devices do not behave as expected when your M Battery Charger is nearby, separate the devices.*

- In the event that you experience any issues with your product, please contact your cochlear implant professional or the manufacturer. Do not attempt to service or modify the Naída CI M, Sky CI M, or its accessories. Doing so may compromise system performance and will void the manufacturer's warranty. Products should be serviced only at Advanced Bionics and damaged products should be returned to Advanced Bionics.*

## **Undesirable Side Effects**

Undesirable side effects of your Naída CI M or Sky CI M system may include skin irritation and discomfort from pressure on the ear, device overheating, or overly loud sounds.

## **Product Description and Performance Characteristics**

The M Batteries are Lithium Ion rechargeable batteries. There are four M Batteries: Small, Medium, Large, and Standard.

*Small*

*Medium*

*Large*

*Standard*

The M Battery Charger contains three M Battery charging bays. The charging system consists of the charger and micro USB power supply cable. The wall adapter is shipped separately from Advanced Bionics.

### **Battery Capacity**

PRODUCT	BATTERY CAPACITY (MAH)
M Battery Small	90
M Battery Medium	150
M Battery Large	220
M Battery Standard	170

## Charger Information

RATED DC INPUT VOLTAGE	RATED DC INPUT CURRENT
5V	500mA

## INDICATIONS FOR USE

The M Batteries and M Battery Charger are external components of the HiResolution Bionic Ear System, which is intended to restore a level of auditory sensation to individuals with severe-to-profound sensorineural hearing loss via electrical stimulation of the auditory nerve.

### Adults

- *18 years of age or older.*
- *sensorineural hearing loss or severe-to-profound unilateral hearing loss.*
- *Post-lingual onset of severe or profound hearing loss.*
- *Limited benefit from appropriately fitted hearing aids, defined as scoring 50% or less on a test of open-set sentence recognition (HINT Sentences).*

### Children

- *12 months through 17 years of age.*
- *Severe-to-profound bilateral sensorineural hearing loss or severe-to-profound unilateral hearing loss.*
- *Use of appropriately fitted hearing aids for at least 6 months in children 2 through 17 years of age, or at least 3 months in children 12 through 23 months of age.*

*The minimum duration of hearing aid use is waived if x-rays indicate ossification of the cochlea.*

- Little or no benefit from appropriately fitted hearing aids. In younger children (< 4 years of age), lack of benefit is defined as a failure to reach developmentally appropriate auditory milestones (such as spontaneous response to name in quiet or to environmental sounds) measured using the Infant-Toddler Meaningful Auditory Integration Scale or Meaningful Auditory Integration Scale or  $\leq 20\%$  correct on a simple open-set word recognition test (Multisyllabic Lexical Neighborhood Test) administered using monitored live voice (70 dB SPL). In older children ( $\geq 4$  years of age), lack of hearing aid benefit is defined as scoring  $\leq 12\%$  on a difficult open-set word recognition test (Phonetically Balanced-Kindergarten Test) or  $\leq 30\%$  on an open-set sentence test (Hearing In Noise Test for Children) administered using recorded materials in the sound field (70 dB SPL).*

## **Intended Purpose**

The M Batteries and M Battery Charger are accessories of an auditory active-implantable system, the HiResolution Bionic Ear system. The HiResolution Bionic Ear system is intended to provide auditory sensation via electrical stimulation of the auditory nerve for individuals with severe to profound bilateral or unilateral sensorineural hearing loss. Severe hearing loss is defined as audiometric thresholds greater than or equal to 70 dB HL, but less than 90 dB HL. Profound hearing loss is defined as audiometric thresholds greater than or equal to 90 dB HL. The external components work together with the implant to bypass the damaged part of the inner ear and

convert sound picked up by the microphone or streamed via wireless communication into electrical signals that are used by the cochlear implant to enable hearing.

M Batteries are intended to power the Advanced Bionics Naída CI M and Sky CI M Sound Processors. The M Battery Charger is intended to recharge only the M Batteries. Placing a charged M Battery on the sound processor will power on the sound processor. Removing the M Battery will power off the sound processor.

### ***Intended Users***

The intended users of the M Batteries and M Battery Charger are users of the Naída CI M or Sky CI M Sound Processors, their caregivers if applicable, and hearing care professionals.

The recipient, or their caregiver, should, at a minimum, be capable of changing the battery, connecting the earhook, placing and removing the sound processor from their ear, and placing and removing the headpiece from the implant site.

The hearing care professional should be trained in the use and fitting of the cochlear implant system.

### ***Intended Use Environment***

The intended use environments of the M Batteries and M Battery Charger are healthcare and daily living environments. The M Batteries are designed to be worn daily during the user's normal waking hours. There is no minimum or maximum duration of use for the M Battery Charger.

## Compatibility

Naída CI M Batteries are compatible with Naída CI M and Sky CI M Sound Processors from Advanced Bionics. The M Battery Charger is compatible with only M Batteries.

## Operating Instructions

### ***General Charging Information for M Batteries***

- The M Battery Charger provided with the sound processor is designed to recharge up to three M Batteries simultaneously. Using the supplied power adapter, it takes approximately 3 to 4 hours to fully recharge a depleted M Battery. If the M Battery is stored for prolonged periods of time, it may take approximately 5 to 6 hours to fully recharge the battery. This will vary slightly for different M Battery types.
- M Batteries do not need to be fully depleted before charging. A partially charged M Battery may be fully charged in a shorter time period.
- You can remove the M Battery from the charger at any time.
- You will not damage the M Batteries if they remain on the charger longer than the required charging time or overnight.
- It is recommended to use and recharge the M Batteries at least once every 3 months. Please note that M Batteries will decrease in capacity with age, even if not in use. This is normal behavior for all rechargeable batteries and should not be considered a defect.

## **Setting up the charger**

### ***To use the M Battery Charger with a wall outlet for charging:***

Place the M Battery Charger on a flat surface.

Connect the micro USB end of the power supply cable into the charger.

Connect the power supply cable to the USB wall adapter.

Plug the wall adapter into the wall outlet.

### ***To use the M Battery Charger with a USB port for charging:***

Place the M Battery Charger on a flat surface.

Connect the micro USB end of the power supply cable into the M Battery Charger.

Insert the USB jack into a USB port (on a computer, in a car, etc.).

## ***Understanding M Battery Charger LED (light-emitting diode) indicators***

The M Battery Charger has different indicator lights that are located at the base of each charging port, as well as where the power cable enters the charger. If the light near the power jack is lit and green in color, it means the charger is connected to a power source and ready to charge the M Batteries. If the light near the power jack is lit and red in color, this indicates a fault condition.\*

The LEDs located at the base of each charging port have 3 color options:

- **Blue: M Battery is charging.**
- **Green: M Battery is fully charged.**
- **Red: Fault condition.\***

\*Try to reinsert the power cable or M Battery. If the red LED illuminates again, contact your cochlear implant professional.

The charger has an automatic dimmer. The brightness of the LEDs may vary depending on the brightness of the room in which the charger is placed.

### ***Inserting the M Batteries into the charger***

Locate the slide tracks on the charger. Position the M Battery so that the battery contact is facing down toward the base of the charger and aligned with the tracks on the charger.

Slide the M Battery along the tracks until it engages. The charger is designed so the M Batteries can only be inserted in one direction.

### ***Removing the M Batteries from the charger***

Gently slide the M Battery out of the slide tracks of the charger, toward the front edge of the charger. Do not pull directly up on the battery.

## **Using the M Batteries with the Naída CI M and Sky CI M Sound Processors**

### **Putting the M Batteries on the sound processor**

Hold the sound processor in one hand. Rotate headpiece cable upward in order to avoid bumping it upon battery placement.

Align the M Battery connector with the sound processor housing side of the connector.

Slide the M battery onto the sound processor until it clicks into place. The sound processor will be powered on as soon as a charged M Battery is fully attached.

**NOTE:** Do not force the M Battery onto the sound processor. The M Battery is designed to be inserted in only one direction; applying force may damage the equipment.

### ***Removing the M Batteries from the sound processor***

Rotate headpiece cable upward in order to avoid bumping it upon M Battery removal.

Hold the sound processor in one hand. In the other hand hold the M Battery. Firmly slide the M Battery cartridge towards the back of the sound processor, away from any earhook accessory.

The sound processor is powered off when the M Battery is removed.

## CARE AND MAINTENANCE

### ***Storing the M Batteries and M Battery Charger***

When not in use, store M Batteries and the M Battery Charger in the equipment case provided.

### ***Cleaning and Maintenance***

Wipe the outside of the M Batteries with a soft, dry cloth. To prevent intermittent operation of the sound processor, the battery contacts on the M Battery Charger, M Battery, and on the sound processor should be kept free from dirt and dust. Gently clean the area with a hearing aid brush or dry cotton swab at least once a month. If the M Battery is exposed to moisture, place it in a drying system before charging and before next use. If the M Battery is dropped, inspect it for evidence of damage or cracking. If any evidence of damage is seen, the M Battery should be replaced.

## Recommended Operating and Storage Temperature Ranges

CONDITION	MINIMUM	MAXIMUM
Operating Temperature	0°C (32°F)	45°C (115°F)
Storage Temperature	-20°C (-13°F)	55°C (131°F)
Relative Humidity	0%	95%

### ***Expected Life***

The expected life of the M Batteries is 1 year.

The expected life of the M Battery Charger is 5 years.

### ***Ingress Protection (IP) Ratings***

The following product configurations have an IP rating of 54 (protection against dust; protection against failure due to water splashes from all directions):

- **The Naída™ CI M90 sound processor with an M Battery or M Zn-Air Battery Pak, an M T-Mic™ microphone or M Acoustic Earhook, and a Slim HP, Slim HP Mic, or Slim HP Standard.**

The following product configuration has an IP rating of 57 (protection against dust; protection against failure due to one-time immersion for 30 minutes to a depth up to 1 meter and after drying overnight in a drying system):

- The Naída CI M90 sound processor with an M Battery, an M Earhook, and a Slim HP, Slim HP Mic, or Slim HP Standard.

## DISPOSAL AND HAZARDOUS MATERIALS

For proper disposal of rechargeable batteries in Europe, please return depleted M Batteries to the nearest AB location or the designated programming/follow-up center. Hazardous materials: not applicable to this product.

In the United States and Canada, please call the Rechargeable Battery Recycling Corporation hotline at 877.273.2925 or visit their website at [www.call2recycle.org](http://www.call2recycle.org).

## AVAILABLE PRODUCT OPTIONS AND MODEL NUMBERS

### *M Battery Charger*

PRODUCT OPTION	MODEL NUMBER
M Battery Charger	CI-5607

### *M Battery Small*

PRODUCT OPTION	MODEL NUMBER
Sand Beige	CI-5551-120
Chestnut	CI-5551-130

PRODUCT OPTION	MODEL NUMBER
Silver Grey	CI-5551-140
Velvet Black	CI-5551-150
Alpine White	CI-5551-110
Lava Red	CI-5551-280
Blue Ocean	CI-5551-260
Majesty Purple	CI-5551-270
Caribbean Pirate	CI-5551-190
Precious Pink	CI-5551-250
Resin Beige	CI-5551-240

### ***M Battery Medium***

PRODUCT OPTION	MODEL NUMBER
Sand Beige	CI-5552-120
Chestnut	CI-5552-130
Silver Grey	CI-5552-140
Velvet Black	CI-5552-150
Alpine White	CI-5552-110

PRODUCT OPTION	MODEL NUMBER
Lava Red	CI-5552-280
Blue Ocean	CI-5552-260
Majesty Purple	CI-5552-270
Caribbean Pirate	CI-5552-190
Precious Pink	CI-5552-250
Resin Beige	CI-5552-240

### ***M Battery Large***

PRODUCT OPTION	MODEL NUMBER
Sand Beige	CI-5553-120
Chestnut	CI-5553-130
Silver Grey	CI-5553-140
Velvet Black	CI-5553-150
Alpine White	CI-5553-110
Lava Red	CI-5553-280
Blue Ocean	CI-5553-260
Majesty Purple	CI-5553-270

PRODUCT OPTION	MODEL NUMBER
Caribbean Pirate	CI-5553-190
Precious Pink	CI-5553-250
Resin Beige	CI-5553-240

### ***M Battery Standard***

PRODUCT OPTION	MODEL NUMBER
Sand Beige	CI-5555-120
Chestnut	CI-5555-130
Silver Grey	CI-5555-140
Velvet Black	CI-5555-150
Resin Beige	CI-5555-240

## **CLINICAL BENEFITS**

The Intended Clinical Benefit of the M Batteries and the M Battery Charger as part of the HiResolution Bionic Ear System is to:

- *provide useful hearing to individuals with severe-to-profound hearing loss via electrical stimulation of the auditory nerve.*

- *provide a combined type of stimulation, electric plus acoustic, when sound processors are enabled with the Acoustic Earhook.*

***Users of the M Batteries and M Battery Charger in the European Union should report any serious incident to their local competent authority and to Advanced Bionics.***

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©2021 Advanced Bionics LLC. All Rights Reserved.
Advanced Bionics LLC